

Mio Move

SPARE PARTS LIST



Dear customer,
in the following pages you will find a complete spare part list with all parts used to build our wheelchairs.

To ensure a accurate delivery of spare parts it is vital to provide the following information fro the identification label in addition to the Article-Number (Art.-No.).

To find the identification label on the chair please look up the following link for the exact location:

http://www.sorgrollstuhltechnik.de/fileadmin/user_upload/pdf/Download/Finding_identification_label.pdf

1. Serial number of the chair

The serial number enables us to identify the configuration of delivery of the specific chair in.

2. Seat width, seat depth and back height

The measures on the identification label are specified for the delivery configuration. If you changed any of these measures after delivery e.g. by growing the chair, please specify the actual new measures.

3. Lower leg length

We need the LLL to decide for the right version of several attachments which are connected to the LLL.

4. Frame colour

Please indicate the frame colour of the chair when ordering a powder coated spare part. Please note also if the finish alternative is shiny or matt.

5. Any other change after delivery (e.g. attachments)

We cannot know about changes after delivery. If you are not sure if such a change has any influence on the needed spare parts, please give us some details about it.

List of abbreviations

Pcs.	piece number
item no	item number to order specific spare part
Q/1	quantity per wheelchair/assembly group
SW	seat width
SD	seat depth
BH	back height
alt.	alternative
l/r	left/right

Handling annotations

Only items with item number are to order.

right / left are in relation to driving direction.

If you need further assistance, please contact us:

SORG Rollstuhltechnik GmbH & Co.KG

Benzstraße 3-5

68794 Oberhausen-Rheinhausen

Fon +49 7254 – 9279 0

Fax +49 7254 – 9279 10

info@sorgrollstuhltechnik.de

<http://www.sorgrollstuhltechnik.de>

Opening times:

Monday – Friday 8.00 am – 4.30 pm

Technical alterations and misprints excluded.

Prices on demand.

Contents

Frame

Frame	2
Anti-tipper	3
Tipping aid	4

Leg support

Leg support standard	5
Leg support, undivided, angle adjustable	6
Leg support, divided, angle adjustable	8
Calf rest	10
Angle-adjustable calf pads for leg support	11
Heel cup	12
Heel edge	13

Seat

Seat plate and seat extension	15
Seat increase	17
Unlocking mechanism seat tilting	18
Seat cushion	20
Abduction wedge	21
Therapy table	22

Back

Back, adjustable in angle / Curved back plate	23
Holder set for curved back plate	26
Seat cover, adjustable	27
Back extension	28
Back cushion	29
stabilizer bar undivided (from back height 35cm)	30
Push handle standard, cranked, adjustable	32
Push bow with synthetic joint	33
Push bow with metal joint	34
Push bow with synthetic joint up to SW30	35
Push bow with metal joint from SW32	36
Side truss pad	37
Head rest	39
Light weight head rest	40
Connection for back shell	42
Sun shade	79

Side part

Side part	43
Side part wheel base extension	45
Arm pad for side guards	47

Contents

Caster wheel

Caster wheels	48
Caster wheel fork	49
Track locking	50
Track fixation	51

Rear wheel

Axle adapter	52
Drum brake anchor	53
Wheel base extension	54
Rear wheels	55
Double handrim Standard	56
Double handrim drum brake	58
Tyres	60
Handrim	61
Handrim covers	62
Spoke cover	63


Brakes

Bowden cable brake pressure piece	64
Bowden cable brake control/non-control sides	65
Bowden cable lock	66
Brake for wheel base extension	67
Companion locking brake	69
Cable for companion locking brake	70
Brake lever for drum brake	71

Options

Bumper	72
Water bottle (without tipping lever)	73
2 point pelvic belt with metal fastener	74
2 point pelvic belt with plastic fastener	75
4 point pelvic belt with metal fastener	76
Chest shoulder belt	77

Frame

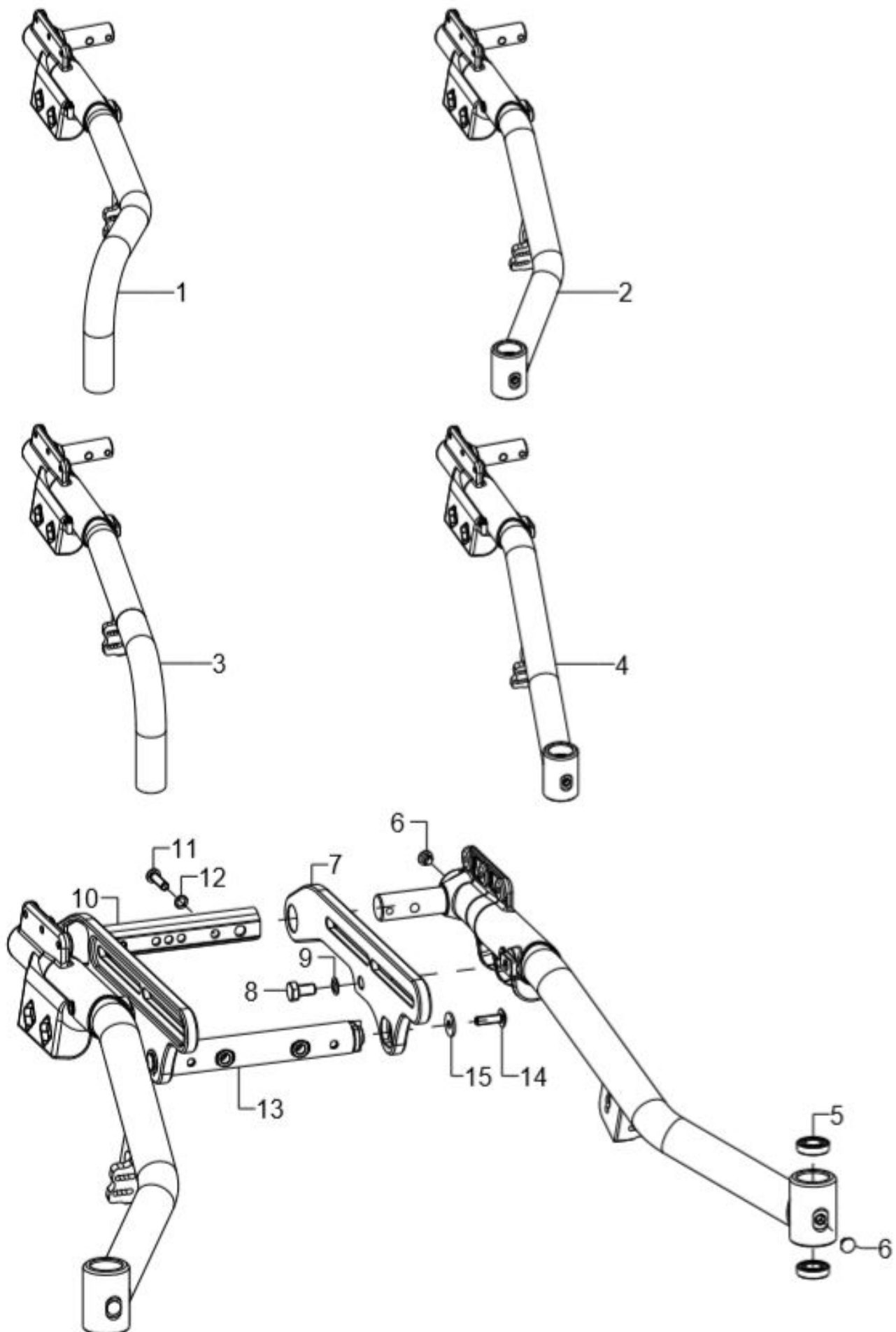




Spare parts list Mio Move

Frame

Page 2 of 79


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1+6	1	59775002			59.77.50.01.02.77
o.B.	1	59775001			59.77.50.01.02.77
1+6	1	59775102			59.77.51.01.02.77
o.B.	1	59775101			59.77.51.01.01.77
1+6	1	59775202			59.77.52.01.02.77
o.B.	1	59775201			59.77.52.01.01.77
2+5-6	1	59774002			59.77.40.01.02.77
o.B.	1	59774001			59.77.40.01.01.77
2+5-6	1	59774102			59.77.41.01.02.77
o.B.	1	59774101			59.77.41.01.01.77
2+5-6	1	59774202			59.77.42.01.02.77
o.B.	1	59774201			59.77.42.01.01.77
3+6	1	59775502			59.77.55.01.02.77
o.B.	1	59775501			59.77.55.01.01.77
3+6	1	59775602			59.77.56.01.02.77
o.B.	1	59775601			59.77.56.01.01.77
3+6	1	59775702			59.77.57.01.02.77
o.B.	1	59775701			59.77.57.01.01.77
4-6	1	59774502			59.77.45.01.02.77
o.B.	1	59774501			59.77.45.01.01.77
4-6	1	59774602			59.77.46.01.02.77
o.B.	1	59774601			59.77.46.01.01.77
4-6	1	59774702			59.77.47.01.02.77
o.B.	1	59774701			59.77.47.01.01.77
5	2		grooved ball bearing ø24/12x6		04.32.12.06.00.10
6	2				04.63.10.07.00.10
o.B.	1	59770104			58.77.01.01.00.77
7-9	1	59770103			58.77.01.01.00.77
8	2				01.04.09.71.02.40
9	2		lock washer S8		02.99.10.80.01.10
10-12	1	58770107			58.77.01.07.00.77
11	2		lens head screw with hexagon socket ISO 7380 - M6x25		01.42.08.56.65.00
12	2		lock washer S6		02.99.10.60.01.10
13-15	1	58770108			58.77.01.08.00.77
14	2		lens head screw with hexagon socket ISO 7380 - M6x20		01.42.08.54.05.00
15	2		washer DIN 9021 -6,4		02.79.06.41.60.20

Anti-tipper

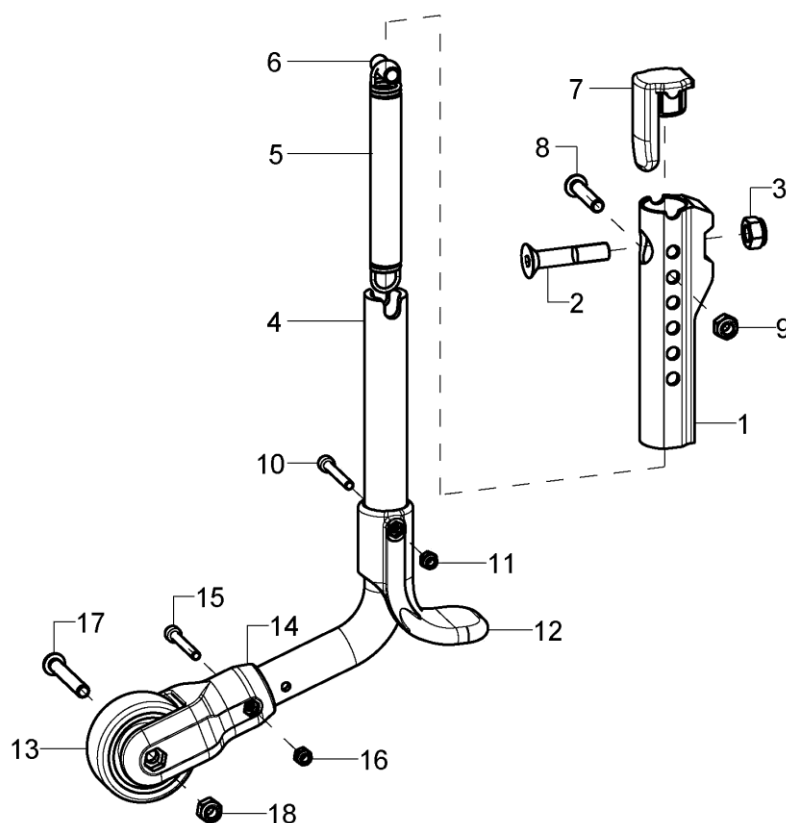


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-18	1	721501	anti-tipper compl., pos. 1 + mounting set A+ anti-tipper bail premounted for 20/22" size driving wheel		72.15.01.02.00.74
1-18	1	721502	anti-tipper compl., pos. 1 + mounting set A+ anti-tipper bail premounted for 24" size driving wheel		72.15.01.03.00.74
1-18	1	721401			72.14.01.01.00.77
1-3	1	721405			72.13.05.01.00.77
2	1				01.65.09.80.02.40
3	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
4-18	1	72150001	anti-tipper bail premounted for 20/22" size driving wheels		72.15.01.99.00.74
4-18	1	72150002	anti-tipper bail premounted for 24" size driving wheels		
4-18	1	721404			72.14.01.99.00.77
5-11	1	72150107	tension spring, loop 180°, 1/1 centred, 13,5x2x113		04.22.01.41.13.00
6	1		spacer bushing 18,4 mm		45.03.10.00.00.73
7	1	721309	locking cap anti-tipper holder		72.13.05.02.00.77
8	1		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
9	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
10	1		cylinder screw DIN 912 M4x30		01.22.07.38.62.40
11	1				02.33.00.40.01.20
12	1	72150104	tipping lever black		72.15.01.04.00.74
13	1	720004	wheel ø50, width 18, nob width 20, axle ø 6, PU grey		04.52.42.50.00.30
14-18	1	72150101	holder anti-tipper for wheel ø50 mm, incl. mounting set		72.15.01.01.00.74
15	1		cylinder screw DIN 912 M4x30		01.22.07.38.62.40
16	1				02.33.00.40.01.20
17	1		lens head screw with hexagon socket ISO 7380 - M6x40		01.42.08.60.65.00
18	1		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20

Tipping aid

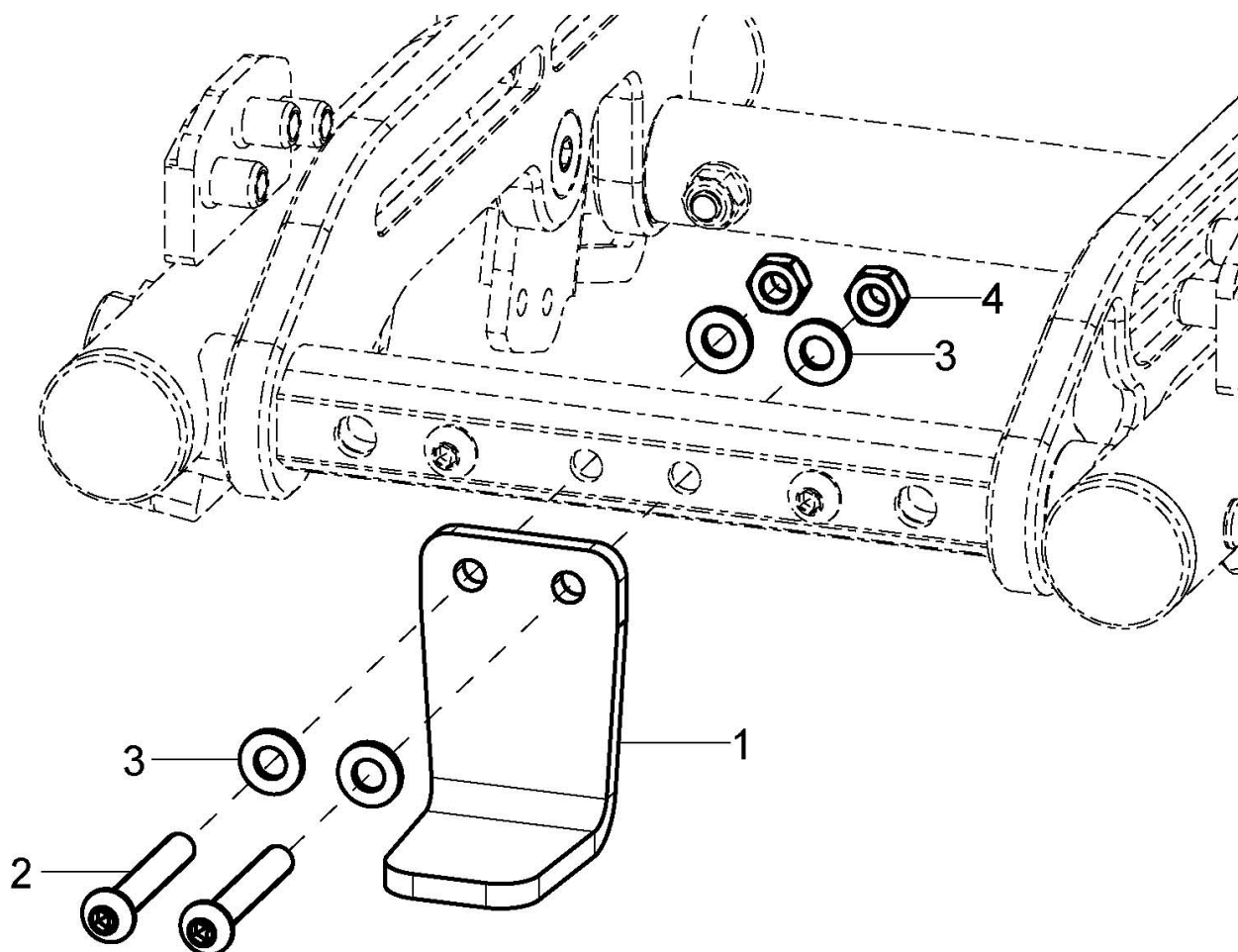


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-4	1	7408			74.06.02.03.00.77
2	2		lens head screw with hexagon socket ISO 7380 - M6x40		01.42.08.60.65.00
3	4		washer DIN 125 - A 6,4		02.61.06.41.41.20
4	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20

Leg support standard

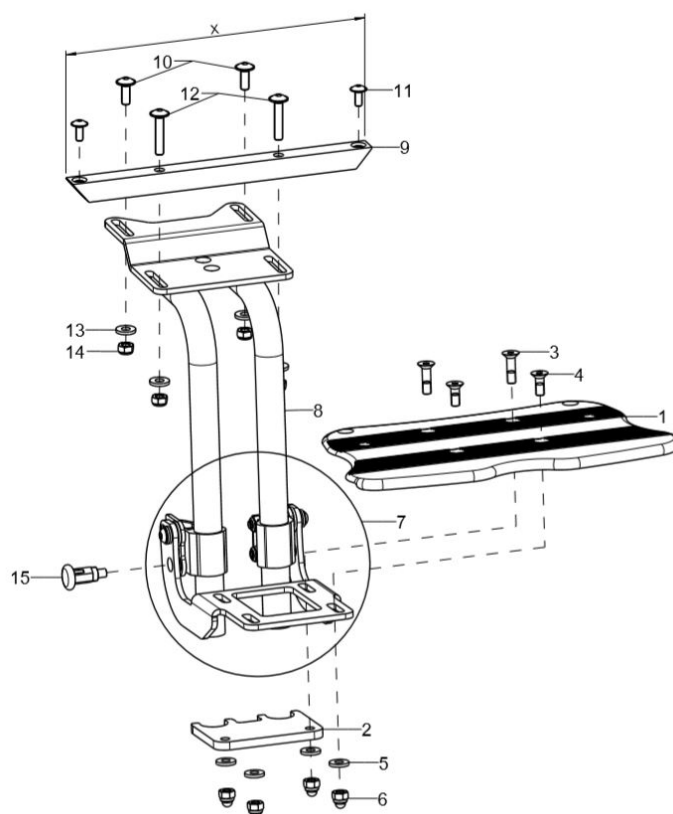

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-14	1	401901	standard leg support, foot plate continuous, folds up to the back, complete		XXXXXXXXXXXX
1	1	40190300	foot plate		40.19.03.08.00.72
2-6	1	40190399	mounting set foot plate		XXXXXXXXXXXX
2	1	40190305	stopper		40.19.03.05.00.72
3	2				01.65.08.55.05.00
4	2		counter sunk screw with hexagon socket DIN 7991 M6x16		01.65.08.52.02.40
5	4		washer reinforced Ø14/6,2x2		58.30.15.01.00.24
6	4		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
7	1	40190304	premounted foot plate holder Kika		40.19.03.99.00.72
8	1	40190101	leg support holder short SD 20-26, to LLL 25 cm		40.19.01.01.00.72
altern.	1	40190102	leg support holder long SD 20-26, to LLL 33 cm		40.19.01.02.00.72
altern.	1	40190103	leg support holder long SD 28-32, to LLL 33 cm		40.19.01.03.00.72
altern.	1	40191001			40.19.10.01.00.78
altern.	1	40191002			40.19.10.02.00.78
altern.	1	40191003			40.19.10.03.00.78
9	1	40190303	leg support crossbar		40.19.03.03.00.72
10-14	1	40190199	mounting set leg support holder		XXXXXXXXXXXX
10	2		lens head screw with hexagon socket M6x16		01.43.08.52.65.00
11	2		lens head screw with hexagon socket M5x12		01.43.07.89.65.00
12	2				01.43.08.58.65.00
altern.	2		lens head screw with hexagon socket M6x35		01.43.08.59.65.00
13	4		washer reinforced Ø14/6,2x2		58.30.15.01.00.24
14	4		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
15	1	690101	latch Ø6/M10x1x31 with lock		04.14.43.60.02.00

Leg support, undivided, angle adjustable

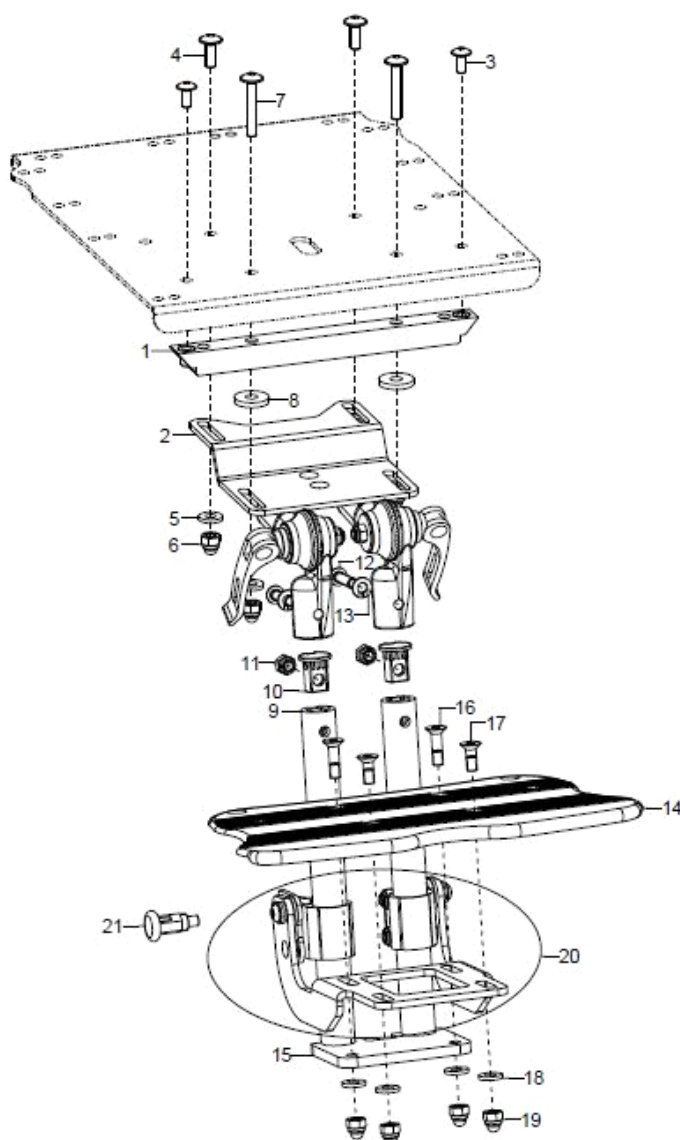

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-21	1	40191005			
1	1	40190303	leg support crossbar		40.19.03.03.00.72
2-8	1	40190701			40.19.07.01.91.77
o.B.	1	40190702			40.19.07.01.92.77
3	2		lens head screw with hexagon socket M5x12		01.43.07.89.65.00
4	2		lens head screw with hexagon socket M6x16		01.43.08.52.65.00
5	4		washer reinforced Ø14/6,2x2		58.30.15.01.00.24
6	4		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
7	2		lens head screw with hexagon socket M6x35		01.43.08.59.65.00
8	2				20.15.16.03.00.17
9-13	2	40190705			40.19.07.07.01.77
o.B.	2	40190706			40.19.07.07.02.77
10	1	653406	plug for nut M6 incl hexagon nut		65.22.20.10.00.00
11	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
12	1		lens head screw with hexagon socket ISO 7380 - M6x25		01.42.08.56.65.00
13	1		spring ring DIN 127- A6		02.85.30.60.05.10
14	1	40190300	foot plate		40.19.03.08.00.72
15-19	1	40190399	mounting set foot plate		XXXXXXXXXXXX




Spare parts list Mio Move

Leg support

15	1	40190305	stopper	40.19.03.05.00.72
16	2			01.65.08.55.05.00
17	2		counter sunk screw with hexogon socket DIN 7991 M6x16	01.65.08.52.02.40
18	4		washer reinforced Ø14/6,2x2	58.30.15.01.00.24
19	4		hexagon nut self-locking low form DIN 985 - M6	02.32.00.60.01.20
20	1	40190304	premounted foot plate holder Kika	40.19.03.99.00.72
21	1	690101	latch Ø6/M10x1x31 with lock	04.14.43.60.02.00

Leg support, divided, angle adjustable

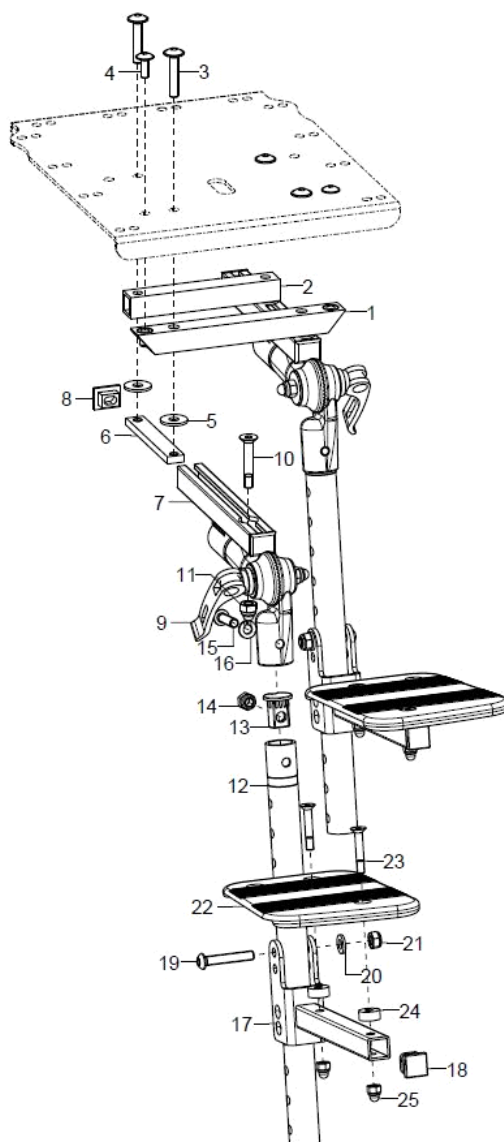

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-25	1	40191105			
1	1	40190303	leg support crossbar		40.19.03.03.00.72
2	1	40190603	tube piece black, for connection leg support incl. mounting set		40.19.06.03.00.72
3-11	2	40190710			
3	4		lens head screw with hexagon socket M6x30		01.43.08.57.65.00
4	3		lens head screw with hexagon socket M5x12		01.43.07.89.65.00
5	4		washer DIN 9021 -6,4		02.79.06.41.60.20
6	2				40.19.07.08.00.77
7+8	2				40.19.07.09.00.77
8	2				35.01.02.05.00.10
9	2	652220	angle adjusting element metal 307° compl with eccentric clamp and mounting set		65.22.20.99.00.10
10	1		counter sunk screw with hexagon socket DIN 7991 M6x35		01.65.08.59.05.00
11	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
12-16	2	40550304			40.55.03.06.01.76
13	1	653406	plug for nut M6 incl hexagon nut		65.22.20.10.00.00
14	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
15	1		lens head screw with hexagon socket ISO 7380 - M6x25		01.42.08.56.65.00




Spare parts list Mio Move

Leg support

16	1		spring ring DIN 127- A6	02.85.30.60.05.10
17-21	2	400103	foot plate joint holder 100720 for continuous foot plate incl. mounting set	40.01.03.02.99.28
18	1		plugs for square tubes, black 15x15	04.63.11.15.00.10
19	1		lens head screw with hexagon socket ISO 7380 - M6x35	01.42.08.59.65.00
20	1		washer DIN 125 - A 6,4	02.61.06.41.41.20
21	1		hexagon nut self-locking low form DIN 985 - M6	02.32.00.60.01.20
22-25	1P	40551300	foot rest 150 mm divided, incl. mounting set	40.55.13.00.00.76
23	2		counter sunk screw with hexagon socket DIN 7991 M5x30	01.65.07.98.05.00
24	2		bushing Ø14/5,2x6	40.55.00.01.00.76
25	2		hexagon nut self-locking low form DIN 985 - M6	02.32.00.60.01.20

Calf rest

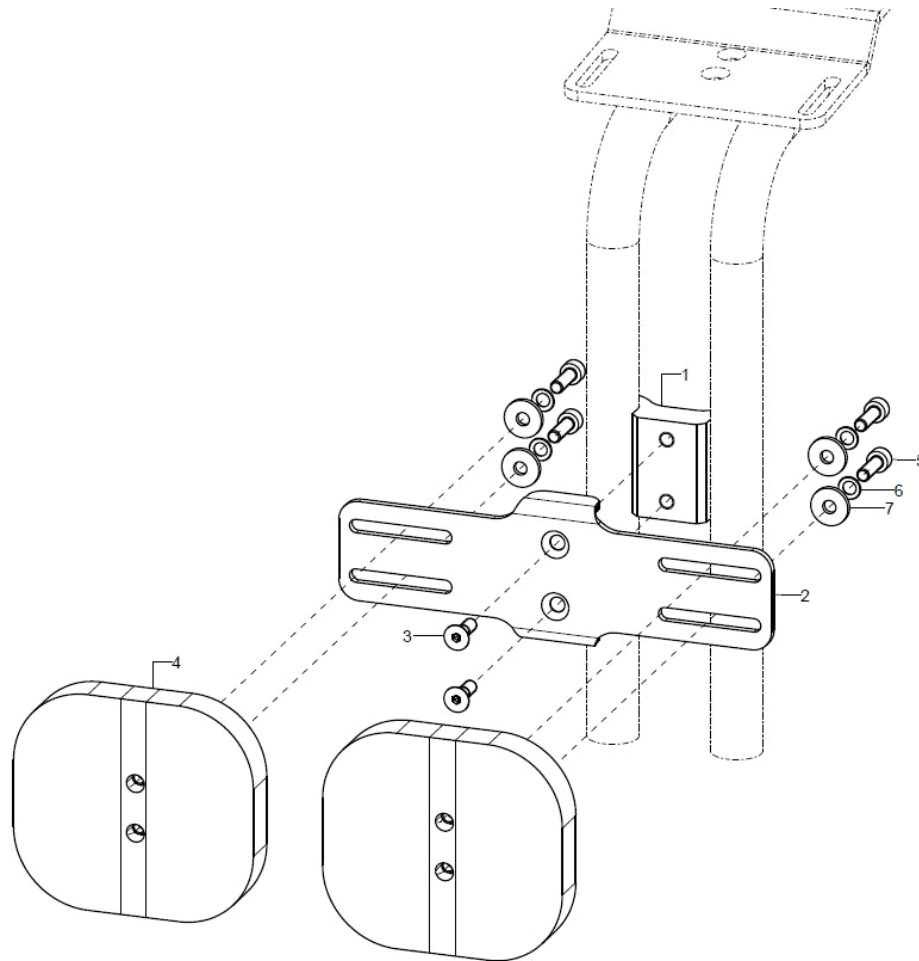


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-7	1	40300801			
altern.	1	40300802			
1-3	1	40190610			
altern.	1	40190611			
1	1				40.19.06.02.00.77
2	1				40.19.06.01.01.77
altern.	1				40.19.06.01.02.77
3	2		counter sunk screw with hexogon socket DIN 7991 M5x16		01.65.07.92.05.00
4-8	2	40300804			40.19.06.04.99.77
altern.	2	40300805			40.30.08.04.99.82
5	2		cylinder head screw DIN 912 M5x16	Bei Halterung anpassbar, Gr.1	01.22.07.92.72.40
6	2		lock washer S5	Bei Halterung anpassbar, Gr.1	02.99.10.50.01.10
7	2		washer DIN 7349 - 5,3		02.77.05.22.00.20
o.B.	2				68.59.01.22.04.00
altern.	2				68.59.01.26.04.00

Angle-adjustable calf pads for leg support


From SW22

From 23cm lower leg length

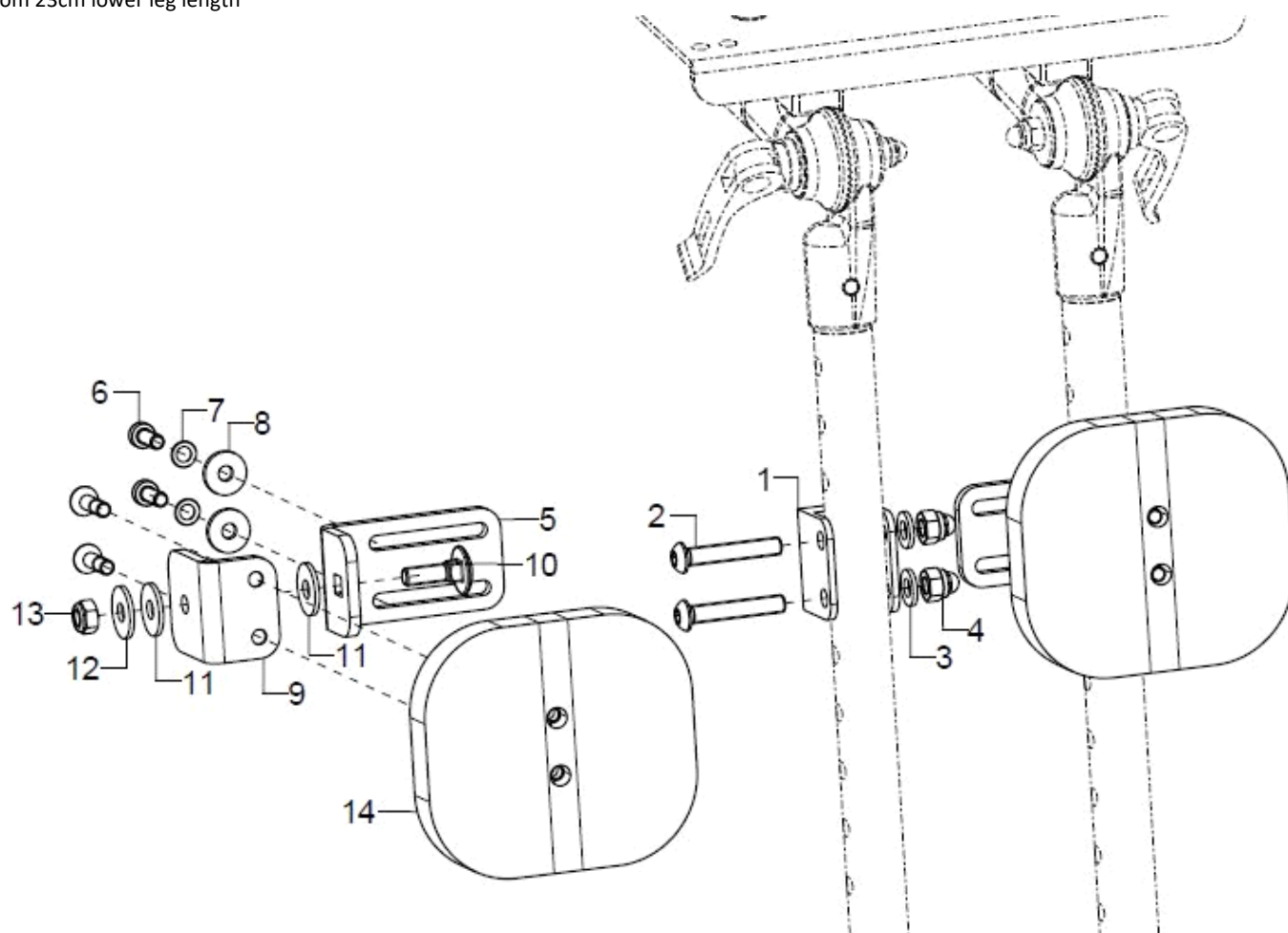


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-14	2	40310700			
1-13	2	40310701			40.19.07.13.00.77
1-4	1	40310705			
2	2		lens head screw with hexagon socket ISO 7380 - M6x35		01.42.08.59.65.00
3	2		washer DIN 125 - A 6,4		02.61.06.41.41.20
4	2		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
5-8	1	40310706			40.19.07.14.00.77
6	2				01.24.07.90.62.40
7	2		lock washer S5	Bei Anbindung Seitenpelotte Gr.1	02.99.10.50.01.10
8	2		washer DIN 7349 - 5,3		02.77.05.22.00.20
9-13	1	40310707			40.19.07.15.00.77
10	1		round-head screw DIN 6003 M6x20		01.11.08.54.00.40
11	2		washer DIN 9021- 6,4- PA		02.79.06.41.60.70
12	1		washer DIN 9021 -6,4		02.79.06.41.60.20
13	1		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
14	1	40300804			40.19.06.04.99.77
o.B.	1				68.59.01.22.04.00

Heel cup

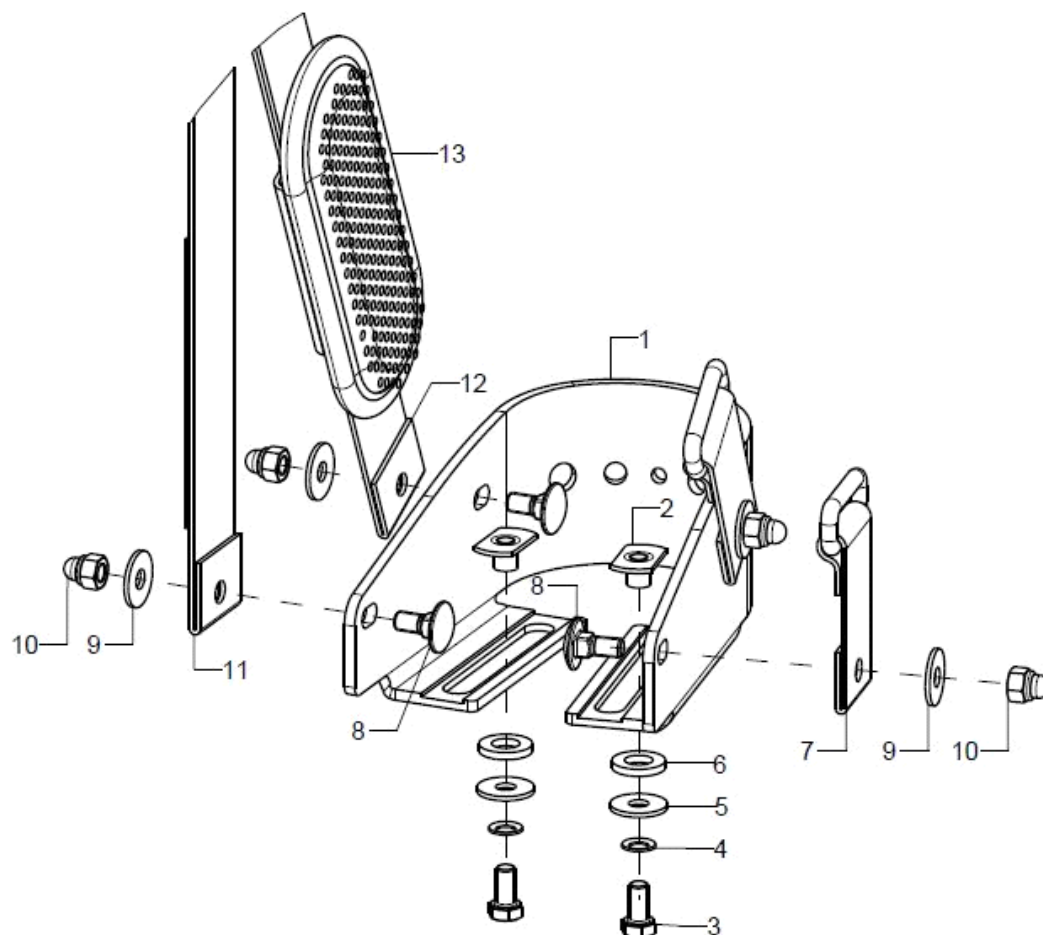


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-6	2	40551501			40.55.15.01.00.76
altern.	2	40551502			40.55.15.02.00.76
altern.	2	40551503			40.55.15.03.00.76
2	1		weld nut C- M5x7,5		02.49.33.03.00.10
3	1				01.04.07.88.02.40
4	1		lock washer S5		02.99.10.50.01.10
5	1		washer DIN 9021 -6,4		02.79.06.41.60.20
6	1				02.83.07.42.00.80
7-13	2	40201501			
altern.	2	40201502			
altern.	2	40201503			

Heel edge

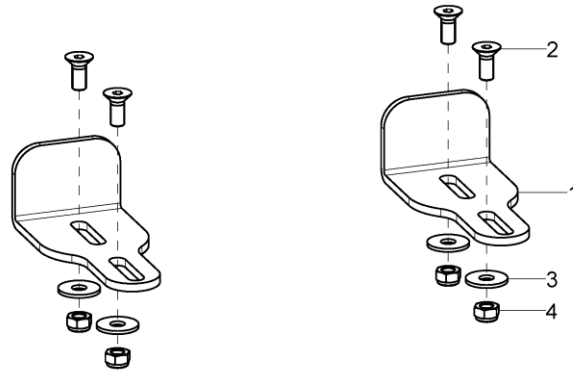


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-4	2	401904	heel edge incl. mounting set		40.19.04.01.00.72
2	2		counter sunk screw with hexagon socket DIN 7991 M5x14		01.65.07.90.05.00
3	4		washer DIN 9021 - M5		02.79.05.31.20.20
4	4		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20

Heel edge

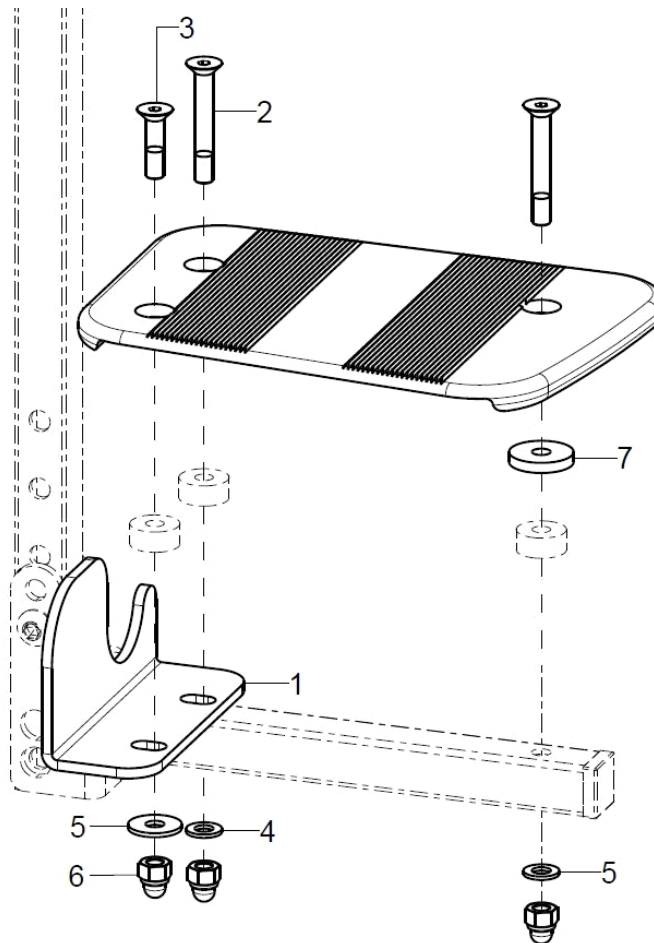


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-7	1P	405509			
1-7	1				40.19.05.01.02.77
o.B.	1				40.19.05.01.01.77
o.B.	1				40.19.05.02.02.77
o.B.	1				40.19.05.02.01.77
o.B.	1	40551012			40.55.10.01.12.76
o.B.	1	40551011			40.55.10.01.11.76
o.B.	1	40551022			40.55.10.01.22.76
o.B.	1	40551021			40.55.10.01.21.76
3	1		counter sunk screw with hexagon socket DIN 7991 M5x35		01.65.07.99.05.00
4	1		counter sunk screw with hexagon socket DIN 7991 M5x20		01.65.07.94.05.00
5	2		washer DIN 125 - A 5,3		02.61.05.31.01.20
6	1		washer DIN 9021 - M5		02.79.05.31.20.20
7	3		hexagon cap nut low form with clamp DIN 986 - M5		02.33.00.60.21.20

Seat plate and seat extension

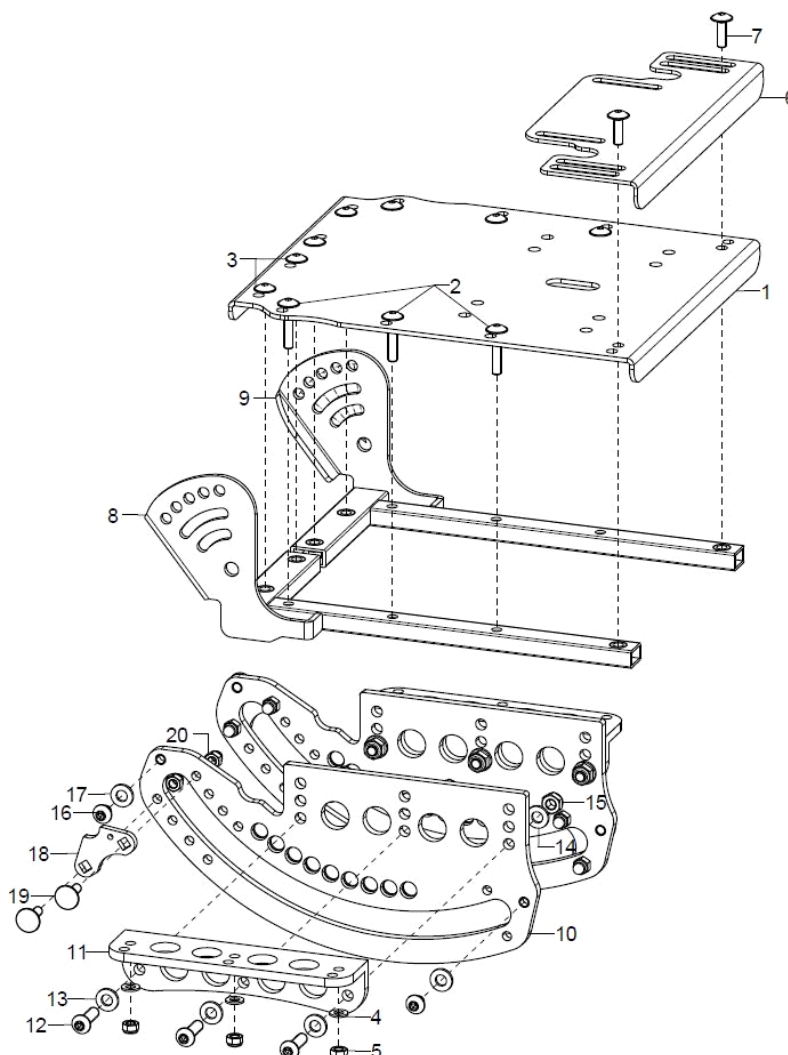


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-5	1	192800			19.28.08.00.00.77
2	6		lens head screw with hexagon socket M5x25		01.43.07.96.65.00
3	4		lens head screw with hexagon socket M5x12		01.43.07.89.65.00
4	6		washer DIN 125 - A 5,3		02.61.05.31.01.20
5	6		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
6+7	1	192106	seat extension incl mounting screws M5x25		19.21.06.01.00.72
7	2		lens head screw with hexagon socket M5x20		01.43.07.92.05.00
8+9	1P	19280100			19.28.01.01.00.77
10	2	19280200			19.28.02.01.00.77
11-17	2				19.28.02.03.00.77
altern.	1P				19.28.02.04.00.77
12	6		lens head screw with hexagon socket ISO 7380 - M6x20		01.42.08.54.05.00
13	3		washer DIN 125 - A 6,4		02.61.06.41.41.20
14	3		washer DIN 125 - A 6,4		02.61.06.41.41.20
15	6		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
16	2				01.42.08.46.65.00
17	2		washer DIN 125 - A 6,4		02.61.06.41.41.20
18-20	2	192810			19.28.06.01.00.77
19	2		round-head screw DIN 6003 M5x16 - 4.6 vz		01.11.07.92.00.40
20	2		hexagon cap nut low form with clamp DIN 986 - M5		02.33.00.60.21.20

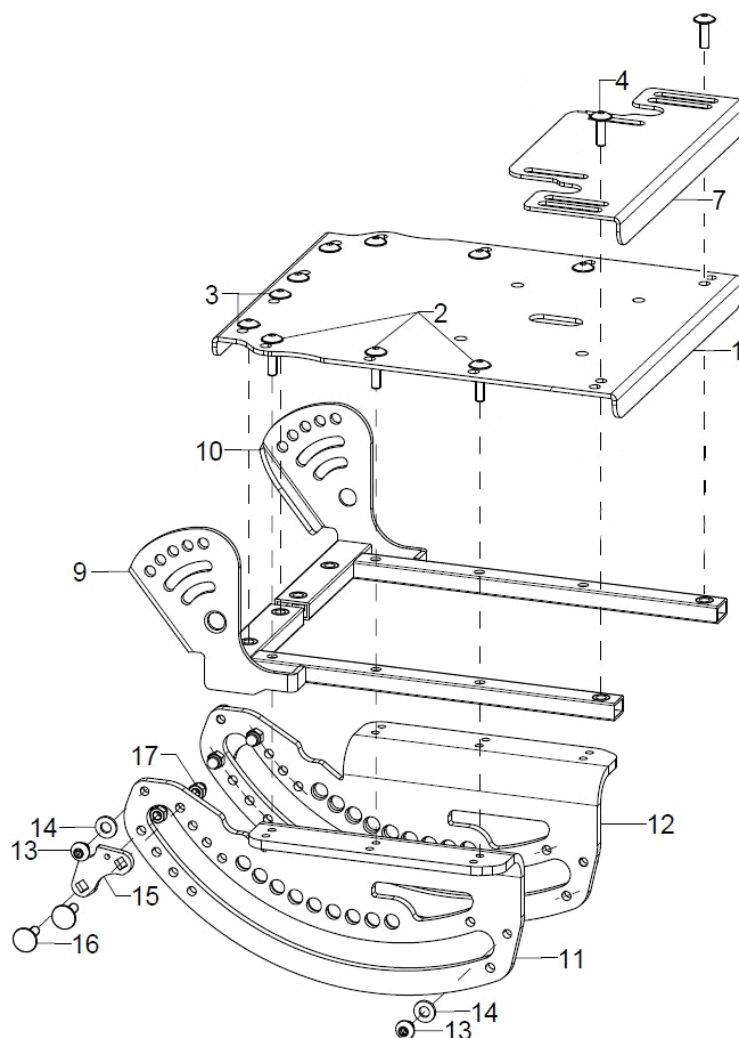


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-4	1	192800			19.28.08.00.00.77
2	6				01.43.07.94.62.60
3	4		lens head screw with hexagon socket M5x12		01.43.07.89.65.00
4	3		lens head screw with hexagon socket M5x20		01.43.07.92.05.00
9+10	1P	19280100			19.28.01.01.00.77
11+13+14	1	19280202			19.28.02.01.02.77
12-14	1	19280201			19.28.02.01.01.77
13	2				01.42.08.46.65.00
14	2		lock washer S6		02.99.10.60.01.10
15-17	2	192810			19.28.06.01.00.77
16	2		round-head screw DIN 6003 M5x16 - 4.6 vz		01.11.07.92.00.40
17	2		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20

Seat increase

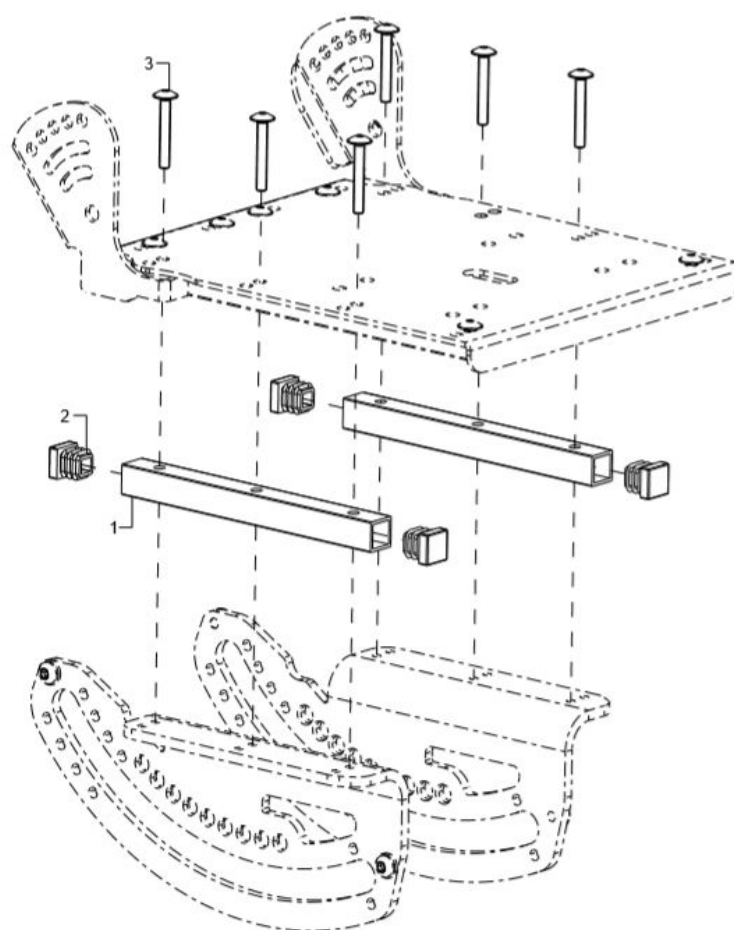

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-3	2	19280701			19.28.07.00.01.77
o.B.	2	19280702			19.28.07.00.02.77
2	1	401107	end plug 15x15 black, 4 piece set		04.63.11.15.00.10
3	3				01.43.07.99.65.00

Unlocking mechanism seat tilting

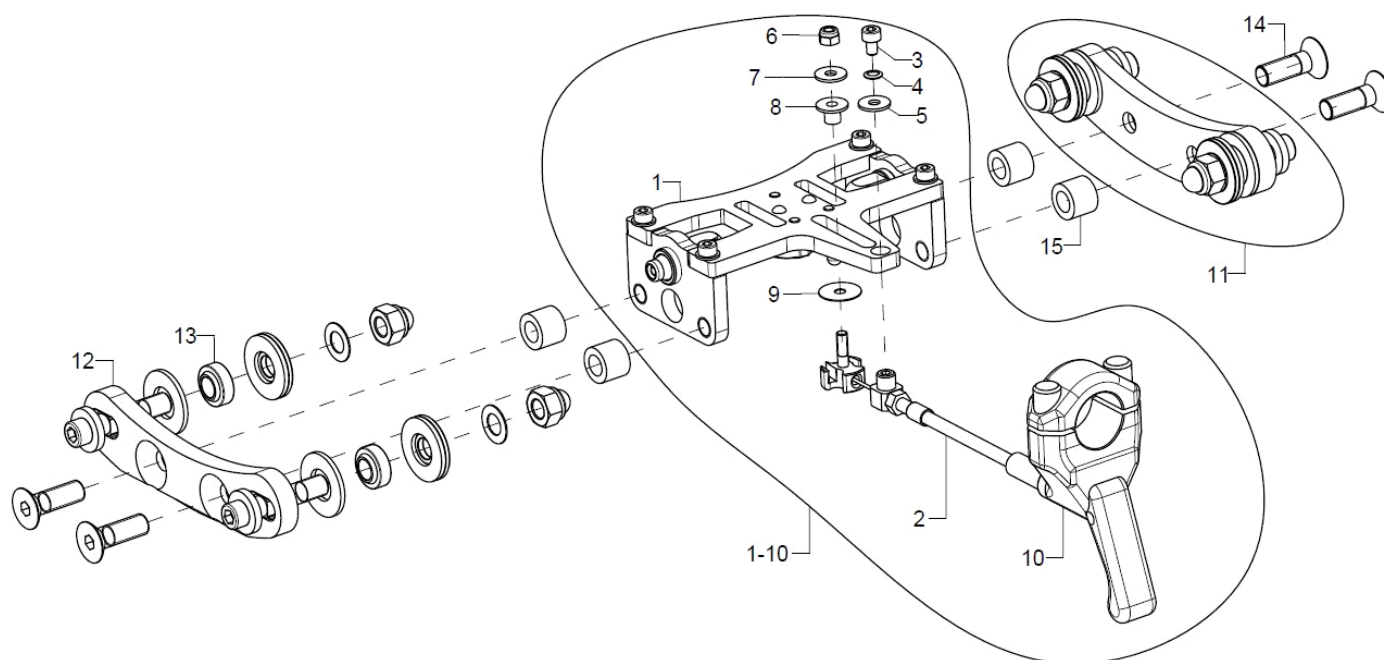

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-10	1	19280400			19.28.04.00.00.77
1	1	19280410			19.28.04.99.00.77
2-10	1	19280416			19.28.04.16.00.77
3	1		cylinder screw DIN 912 M4x6		01.22.07.26.62.40
4	1		lock washer S4		02.99.10.40.01.10
5	1		washer DIN 9021 - M4		02.79.04.31.00.20
6	1		hexagon nut self-locking low form DIN 985 - M4		02.32.00.40.01.20
7	1		washer DIN 9021 - M4		02.79.04.31.00.20
8	1				19.28.04.20.00.77
9	1				19.28.07.10.00.77
10	1	190513	hand release lever for gas pressure spring, not lockable		04.54.92.12.20.10
11-15	2	19280300			19.28.03.01.00.77
12	2				19.28.03.04.00.77
13	4				04.32.08.09.00.00
14	2				01.65.09.77.05.00
15	2				19.28.04.09.00.77






Seat cushion

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
o.B.	1	891002	anatomical seat cushion		XXXXXXXXXXXXX
o.B.	1	891001			
o.B.	1	891003	cover for anatomical seat cushion		XXXXXXXXXXXXX
o.B.	1	891004	incontinence cover for anatomical seat cushion		XXXXXXXXXXXXX
o.B.	1	8003	seat cushion, synthetic foam, 3cm thick		XXXXXXXXXXXXX
o.B.	1	8005	seat cushion, synthetic foam, 5cm thick		XXXXXXXXXXXXX
o.B.	1	8103	seat cushion, memory foam, 3cm thick		XXXXXXXXXXXXX
o.B.	1	8105	seat cushion, memory foam, 5cm thick		XXXXXXXXXXXXX
o.B.	1	800050			
o.B.	1	8603	incontinence protection for 3cm cushion (non refundable)		
o.B.	1	8605	incontinence protection for 5cm cushion (non refundable)		


Stoffmuster	XX
schwarz	30
grau	31
nachtblau	32
ultramarinblau	33
petrol	34
applegreen	35
bordeauxrot	36
rot	37
pink	38

Abduction wedge

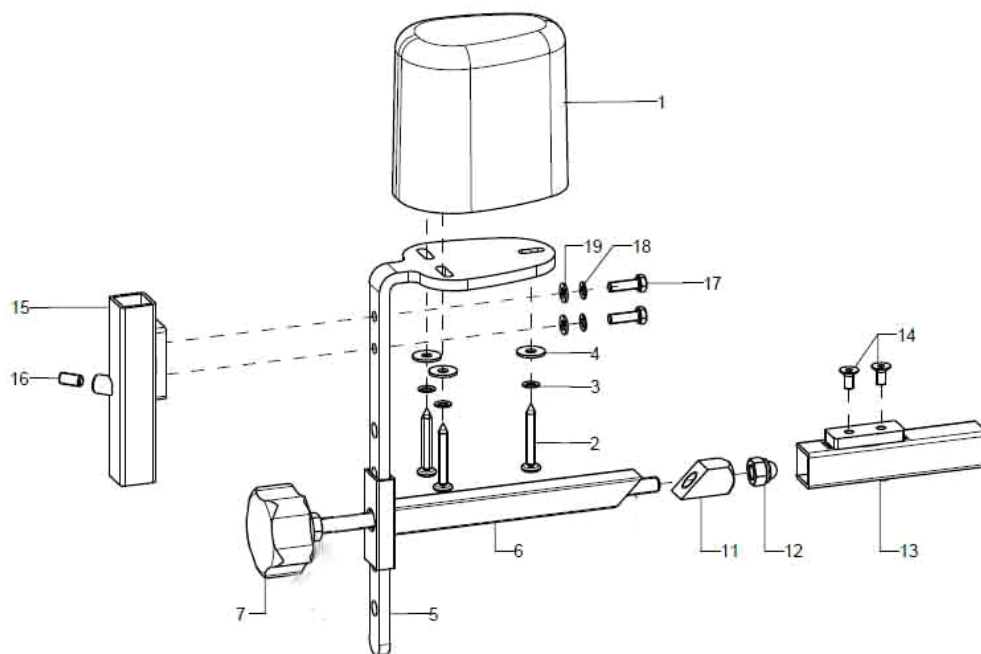


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-14	1	1952	abduction wedge size s		XXXXXXXXXXXX
o.B.	1	1953	abduction wedge size m		XXXXXXXXXXXX
o.B.	1	1954	abduction wedge size l		XXXXXXXXXXXX
1-4	1	195002	abduction wedge small		04.65.17.09.01.20
o.B.	1	195003	abduction wedge medium		04.65.17.11.03.20
o.B.	1	195004	abduction wedge large		04.65.17.13.02.20
2	3				01.88.07.99.00.80
3	3		lock washer S5		02.99.10.50.01.10
4	3		washer DIN 9021 - M5		02.79.05.31.20.20
5	1	195101	bracket for abduction wedge		19.50.03.02.00.10
6	1	195103	weldment clamber		19.50.02.01.00.10
7	1				19.05.03.01.99.82
11	1				68.40.04.04.00.51
12	1				02.33.00.80.41.20
13-14	1	403111	leg support adapter weldment / universal incl. mounting set		40.01.01.05.00.28
14	2		counter sunk screw with hexagon socket DIN 7991 M5x10		01.65.07.88.05.00
15-19	1	400105	universal leg support socket for combination abduction wedge with therapy table incl. mounting set		40.01.01.05.00.28
16	1		setscrew with hexagon socket DIN 913, M6x12		01.67.08.49.61.20
17	2		hexagon screw DIN 933 - M5x16		01.04.07.92.02.40
18	2		lock washer S5		02.99.10.50.01.10
19	2		washer DIN 125 - A 5,3		02.61.05.31.01.20

Therapy table

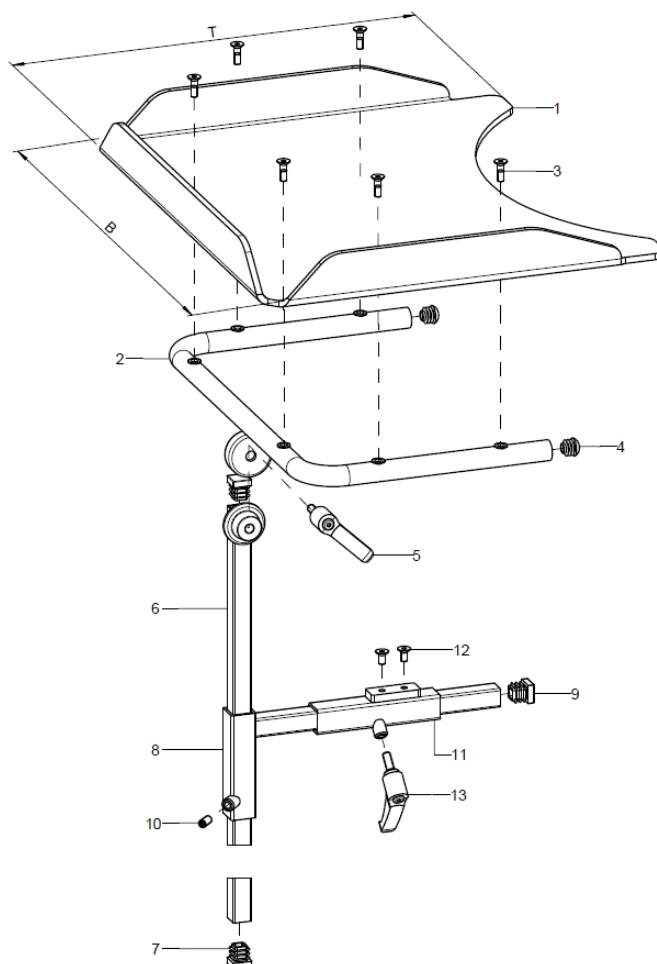

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-13	1	9209XX	therapy table central mounting under SeatFix complete		XXXXXXXXXXXX
1	1	920016	therapy table size 0 (B39xT33) polycarbonate		92.00.16.01.00.72
altern.	1	920030	therapy table size 1 (B45xT36) polycarbonate		92.00.10.01.00.51
altern.	1	920031	therapy table size 2 (B59xT45) polycarbonate		92.00.11.01.00.51
2-13	1	920909	mounting set therapy table for centre attachment (U-holder, cone rod with clamp lever, adapter holder, square adapter)		
2-5	1	920901	holder for therapy table incl. mounting set		92.09.01.01.01.72
o.B.	1	920904			92.09.01.01.02.72
3	6		counter sunk screw with hexagon socket DIN 7991 M5x16		01.65.07.92.05.00
4	2		plugs for tube ø22x1-2, black		04.63.10.22.00.10
5	1	650050	clamp lever adjustable M8x16, black		04.13.10.80.16.50
6-7	1	920902	cone rod 450 incl. plugs		92.08.03.04.00.24
7	2		plugs for square tubes, black 15x15		04.63.11.15.00.10
8-10	1	920802	adapter 220x85 incl. mounting set		92.08.02.01.00.24
9	1		plugs for square tubes, black 15x15		04.63.11.15.00.10
10	1		setscrew with hexagon socket DIN 913, M6x12		01.67.08.49.61.20
11-13	1	400105	universal leg support socket for combination abduction wedge with therapy table incl. mounting set		40.01.01.05.00.28
12	2		counter sunk screw with hexagon socket DIN 7991 M5x10		01.65.07.88.05.00
13	1	45320303			04.13.10.60.16.50



Back, adjustable in angle / Curved back plate

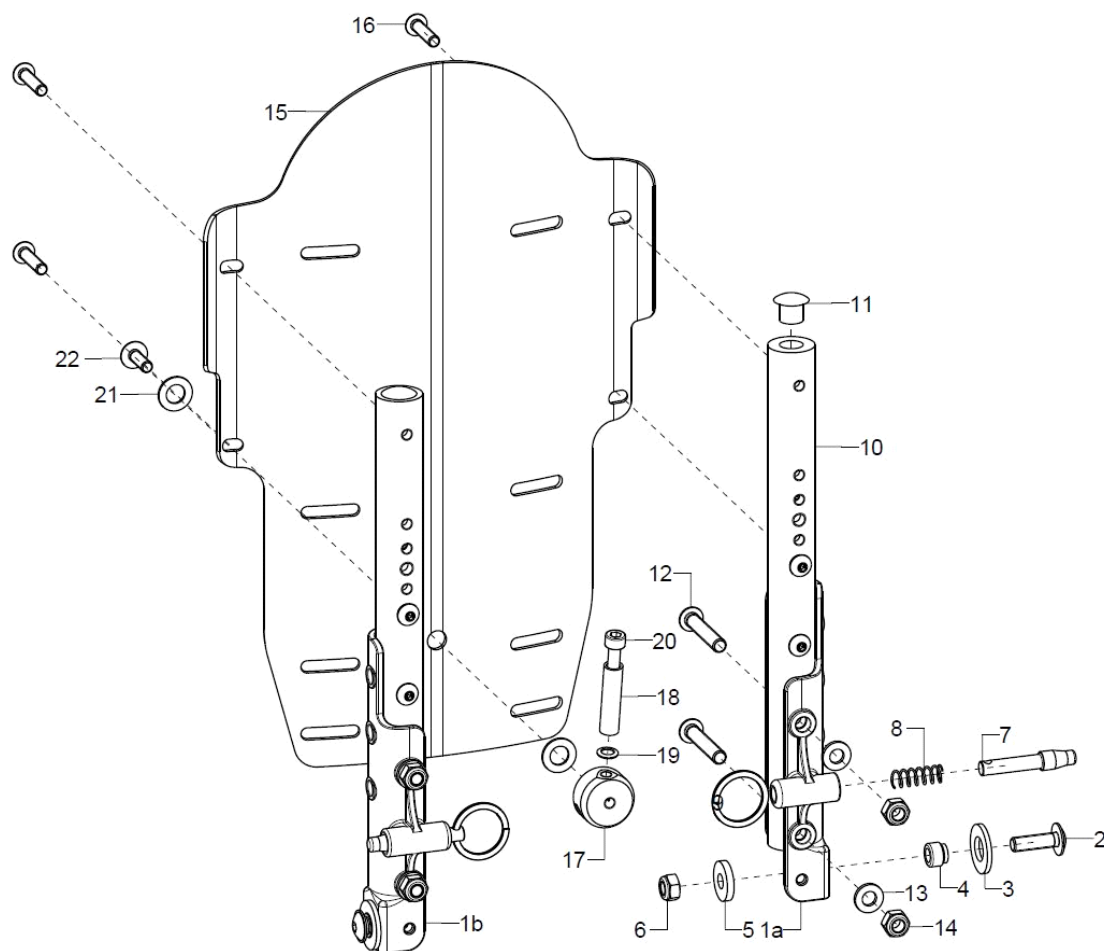


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1a-9	1	62370204	U-part 137 mm, right		61.09.05.01.02.00
1b-9	1	62370203	U-part 137 mm, left		61.09.05.01.01.00
2	1		lens head screw with hexagon socket M6x20		01.43.08.54.65.00
3	1		washer DIN 125 - A 10,5 - PA black		02.61.10.52.01.71
4	1				61.03.02.01.00.18
5	1		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
6	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
7-9	2	610109	locking bolt incl. mounting set (spring and ring)		61.01.02.04.00.24
8	1		pressure spring DIN 2098 0,7x7,65/9,05x21		04.21.00.80.21.00
9	1		wire form ring 2x16		04.29.04.42.20.00
10+11	2	623302XX			62.33.08.01.00.00
11	1				04.63.10.20.01.10
12-14	2	62330099	mounting set back tube		XXXXXXXXXXXX
12	2		lens head screw with hexagon socket ISO 7380 - M6x35		01.42.08.59.65.00
13	2		washer DIN 125 - A 6,4		02.61.06.41.41.20
14	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
15-16	1	6950XX			69.50.XX.00.00.00
o.B.	1	6951XX			69.51.XX.00.00.00
o.B.	1	6952XX			69.52.XX.00.00.00
o.B.	1	6953XX			69.53.XX.00.00.00
o.B.	1	6954XX			69.54.XX.00.00.00
16	4				01.42.07.95.65.00
17-22	1	69730403	lever complete		XXXXXXXXXXXX



17	1			69.73.04.03.00.72
18	1			19.08.01.01.00.69
19	1	lock washer S5		02.99.10.50.01.10
20	1	cylinder screw DIN 912 M5x40		01.22.08.00.62.40
21	2	fender disc 8,4x16x1 PE		02.81.16.00.10.70
22	1	lens head screw with hexagon socket M5x20		01.43.07.92.05.00
o.B.	1	69730404	polyester band 3 mm	00.57.03.05.00.50

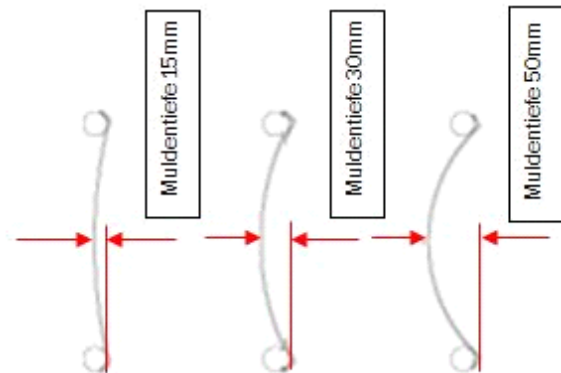
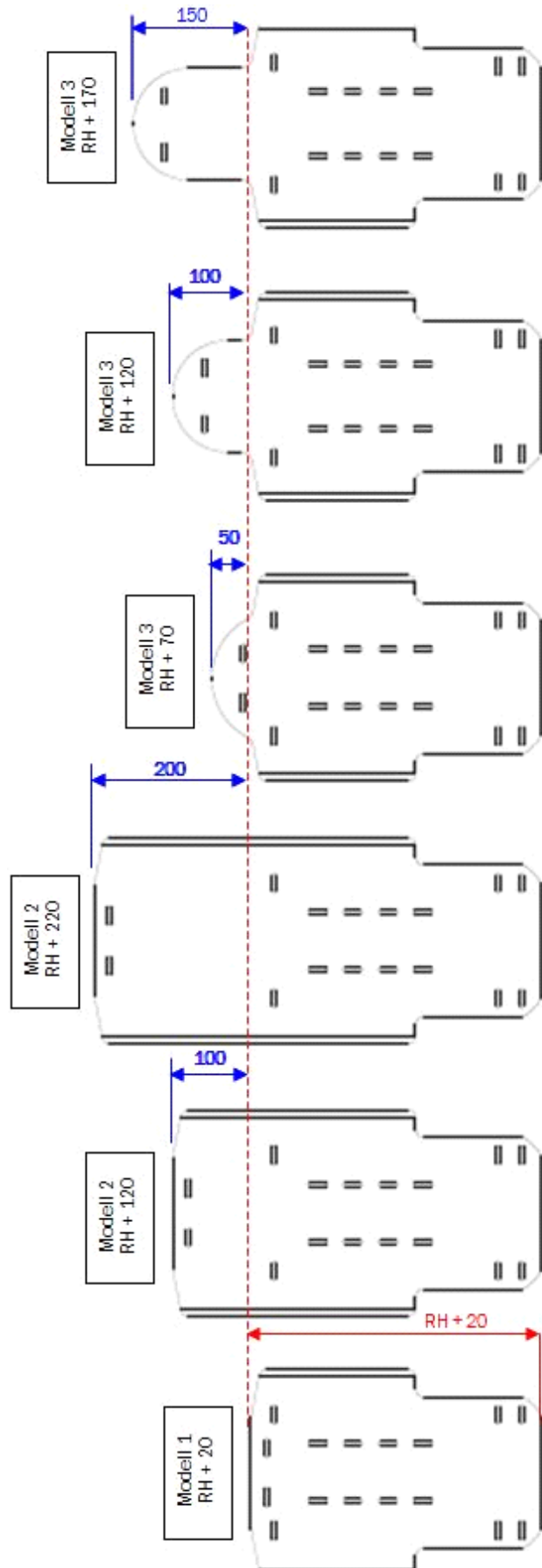


Holder set for curved back plate

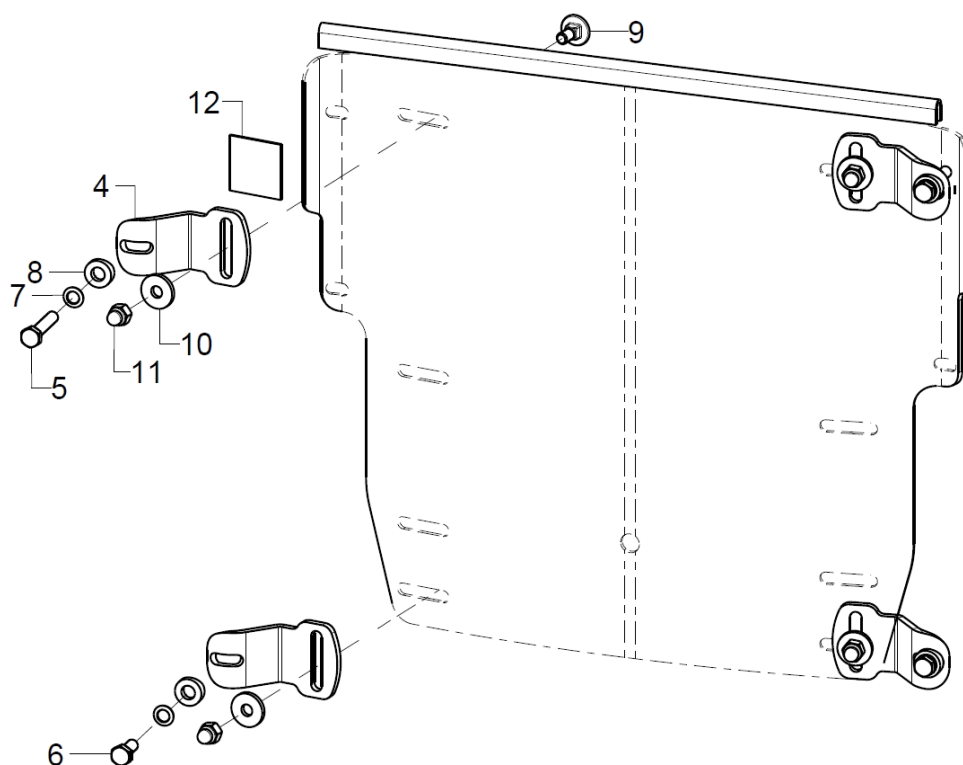

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
4+5+7-12	4	692598	mounting set curved back plate		XXXXXXXXXXXX
4	1	692510	screwed on holding plate		69.25.05.01.00.69
5	1		hexagon screw DIN 933 - M5x25		01.04.07.96.02.40
6	1		hexagon screw DIN 933 - M5x12		01.04.07.89.02.40
7	1		lock washer S5		02.99.10.50.01.10
8	1				40.04.16.01.00.10
9	1		round-head screw DIN 6003 M5x12		01.11.07.89.00.40
10	1		washer DIN 7349 - 5,3		02.77.05.22.00.20
11	1		hexagon cap nut low form with clamp DIN 986 - M5		02.33.00.60.21.20
12	2		Velcro strap 25 mm		00.42.61.02.50.11

Seat cover, adjustable

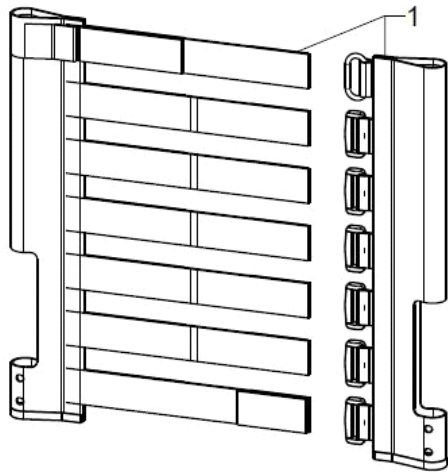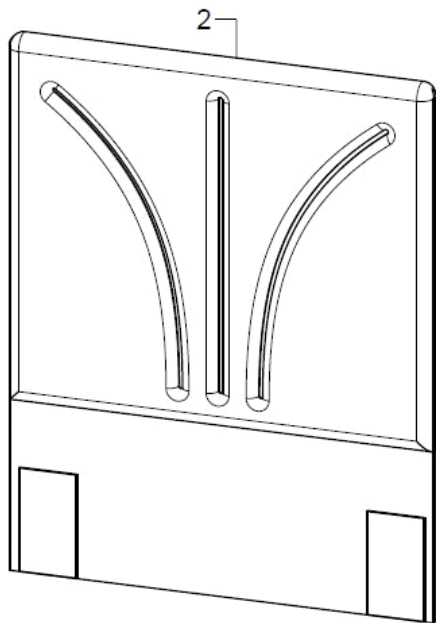

Farbe	XX
schwarz	30
grau	31
nachtblau	32
ultramarinblau	33
petrol	34
applegreen	35
bordeauxrot	36
rot	37
pink	38


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1+2	1	670103XX			XX.XX.XX.XX.XX
1	1	670500	back cover adjustable		XXXXXXXXXXXX
2	1	670599	back cover with cushion 2 cm		XX.XX.XX.XX.XX
o.B.	2		counter sunk screw with hexogon socket DIN 7991 M5x16		01.65.07.92.05.00
o.B.	2		rosette ø13/6x2, black		04.14.70.60.02.00

Back extension

Curved back version 1: only from BH 35 cm

Curved back version 3: only from BH 30 cm, only with BH+5cm

If the curved back is attached with the fitting set for solid back, a back extension is only possible from SW28.

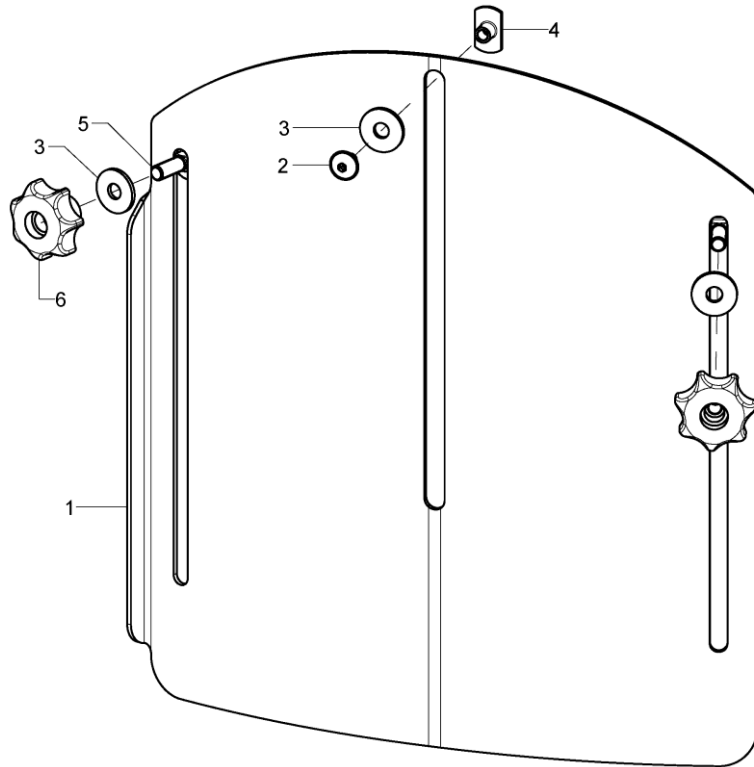


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-6	1	686092			68.60.02.00.00.00
2-6	1	68607299	mounting set back extension		XXXXXXXXXXXX
2	1		lens head screw with hexagon socket M5x8		01.43.07.87.62.60
3	3		washer DIN 9021- 6,4- PA		02.79.06.41.60.70
4	1		weld nut C- M5x7,5		02.49.33.03.00.10
5	2		round-head screw DIN 6003 M6x20		01.11.08.54.00.40
6	2		7-star knob ø32 - M6 internal thread continuous, black		04.05.37.64.20.30



Back cushion

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
o.B.	1	8517	back cushion 3cm with zipper		XXXXXXXXXXXXX
o.B.	1	851750			
o.B.	1	891104	anatomical back (3cm) incl. changeable cover		
o.B.	1	891106	anatomical back (5cm) incl. changeable cover		
o.B.	1	891150			
o.B.	1	68607201	head rest pad incl. cover for back extension		


Stoffmuster	XX
schwarz	30
grau	31
nachtblau	32
ultramarinblau	33
petrol	34
applegreen	35
bordeauxrot	36
rot	37
pink	38

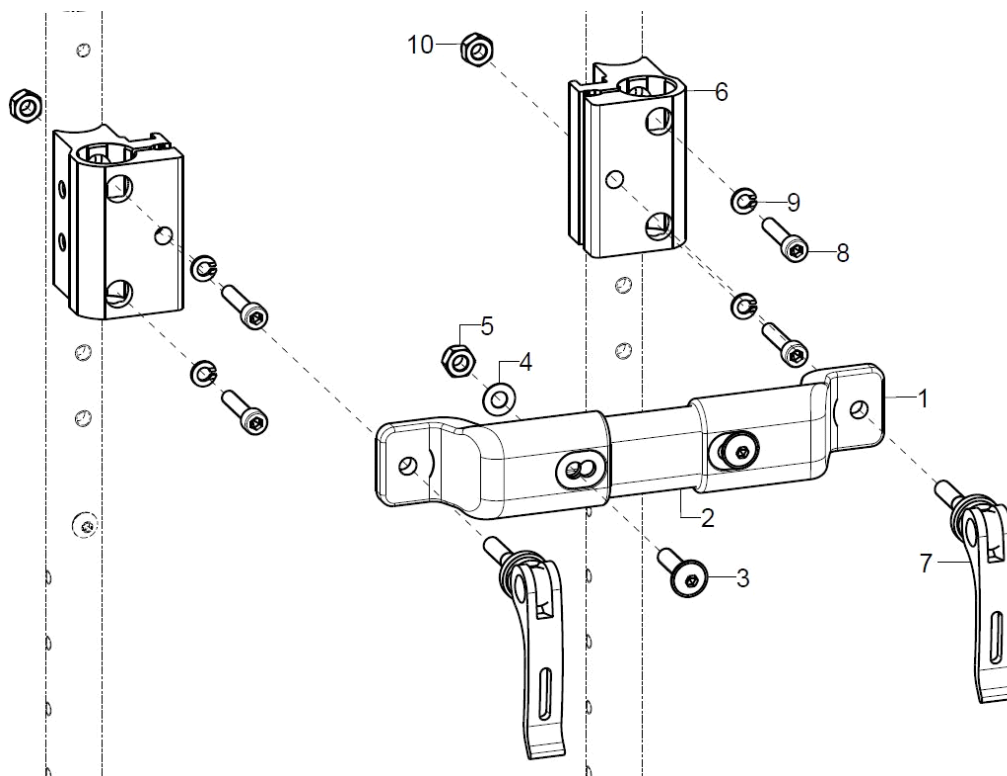stabilizer bar undivided (from back height 35cm)


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-5	1	64090302			
o.B.	1	64090402			
1	2	64090103			64.09.01.03.00.00
o.B.	2	64090106			64.09.01.06.00.00
2-5	1	64090110			64.09.01.07.00.00
o.B.	1				64.09.01.08.00.00
3	2		lens head screw with hexagon socket M6x25		01.43.08.56.65.00
4	2		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
5	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
6-10	2	650061	push bail holder R10		65.00.51.02.00.10
altern.	2	650051	push handle clamp holder 38x42x58, incl. eccentric clamp and mounting set		65.00.51.01.00.10
altern.	2	650065			65.00.56.05.00.77
altern.	2	650066			65.00.56.06.00.77
altern.	2	650064			65.00.56.04.00.69
altern.	2	650056			65.00.56.02.00.18
altern.	2				65.00.56.07.00.77
altern.	2				65.00.56.08.00.77
7	1	652225			04.13.40.60.35.10
8	2		cylinder screw DIN 912 M5x25		01.22.07.96.62.40
9	2		spring ring DIN 127 - A 5		02.85.30.50.05.10
10	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20

SB	L Stabilisierungsrohr
20	55*
22	75*
24	95
26	115
28	135
30	155
32	175
34	195
	*= <u>Stabirohr/Stabiadapter</u> für kleine Sitzbreiten


Push handle standard, cranked, adjustable

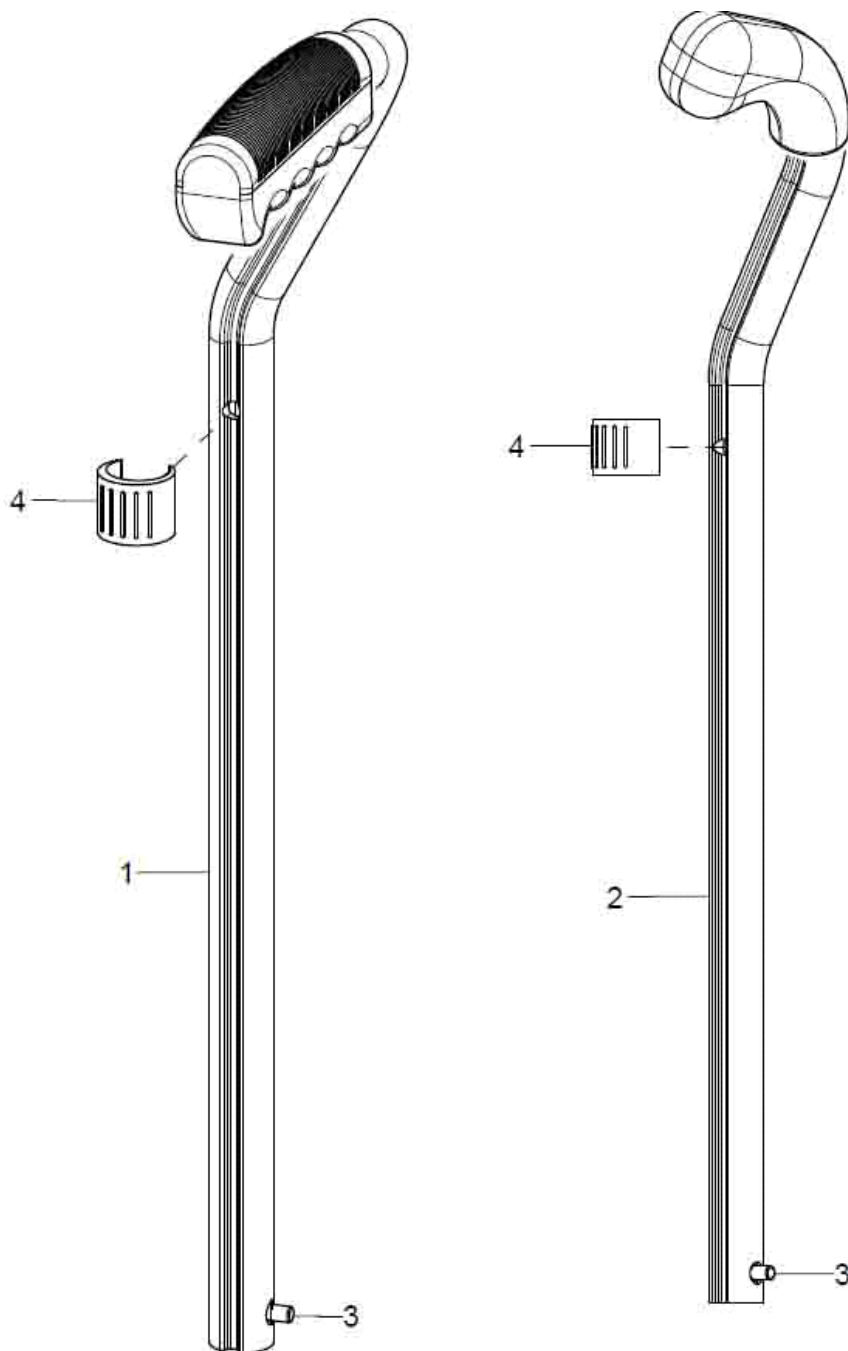

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1+3+4	1	650632	push handle 25 cm, cranked, extendable, right complete		65.06.30.01.06.28
1+3+4	1	650631	push handle 25 cm, cranked, extendable, left complete		65.06.30.01.05.28
1+3+4	1	651702			65.07.30.01.06.28
1+3+4	1	651701			65.07.30.01.05.28
3	2	650160	locking unit for push handles		65.01.30.03.00.10
4	2	046350200010	clamp conductor black, ø 18-20		04.63.50.20.00.10

Push bow with synthetic joint


At SW20/22 and BH25/30 only limited height adjustment possible

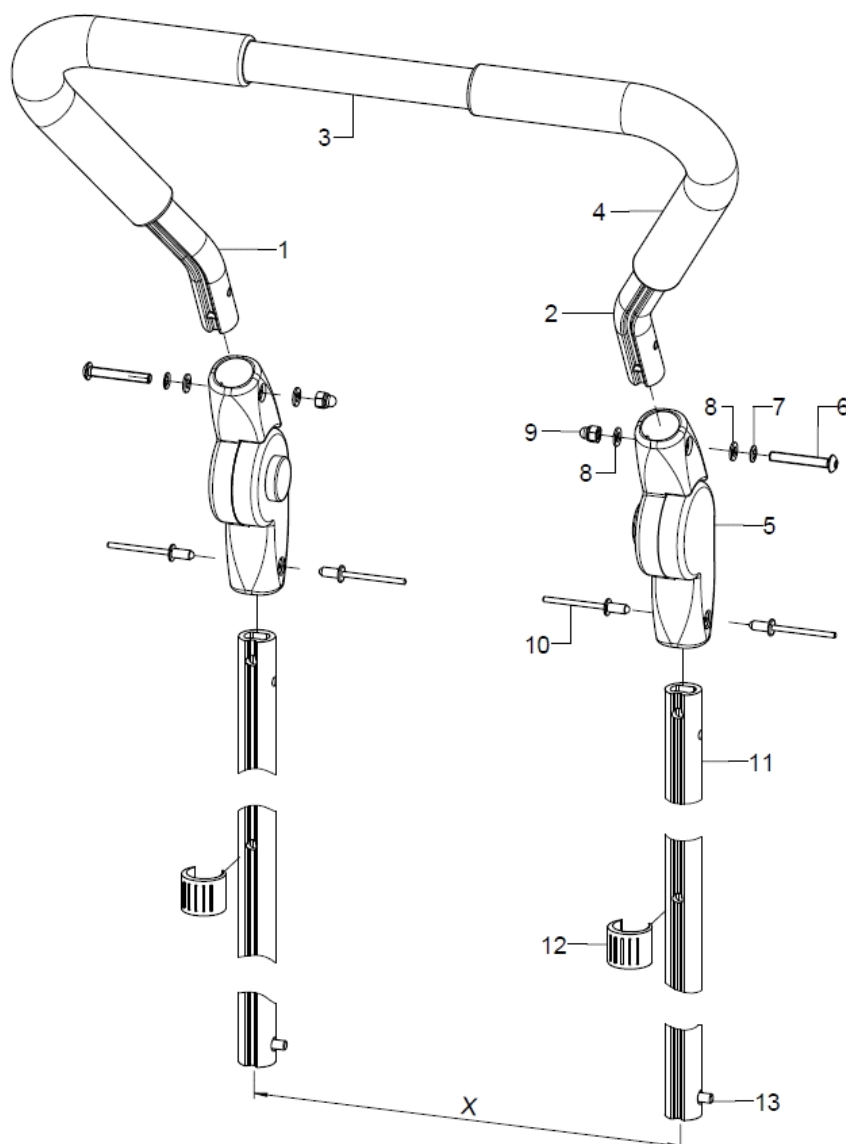

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-13	1	6534	push bail extendable by 22 cm, 205° angle adjustment synthetic joint, complete		XXXXXXXXXXXX
1-4	1	653002	push bail cropped pre-assembled		XX.XX.XX.XX.XX.XX
1	1	653006	push bail left part		65.30.01.02.01.47
2	1	653007	push bail right part		65.30.01.02.02.47
3	1	653004	steel tube incl. riveting		65.30.01.04.00.24
4	2	653005	tube cover - rubber 260 mm		65.30.03.01.00.47
5-10	2	653403	synthetic joint with push knob 205° incl. mounting set		04.03.01.20.01.00
6	1		lens head screw with hexagon socket ISO 7380 - M5x35		01.42.07.99.65.00
7	1		lock washer S5		02.99.10.50.01.10
8	2		washer DIN 125 - A 5,3		02.61.05.31.01.20
9	1		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
10	2		blind rivet F5x10 A2/A2		03.64.00.50.10.30
11-13	2	653202	profile tube support 200		65.31.02.01.00.24
12	1	046350200010	clamp conductor black, ø 18-20		04.63.50.20.00.10
13	1	650160	locking unit for push handles		65.01.30.03.00.10

Push bow with metal joint


At BH25/30 only limited height adjustment possible

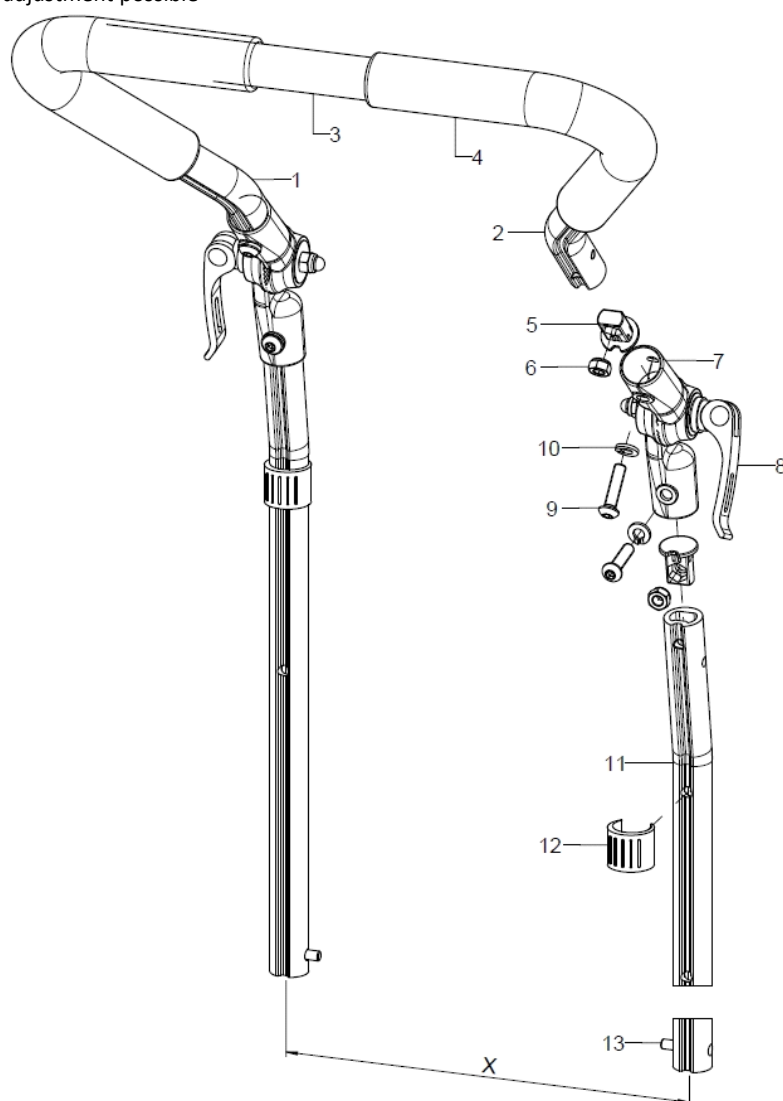

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-13	1	6532	push bail extendable by 20 cm, 307° angle adjustment metal joint, complete		XXXXXXXXXXXX
1-4	1	653002	push bail cropped pre-assembled		XX.XX.XX.XX.XX
1	1	653006	push bail left part		65.30.01.02.01.47
2	1	653007	push bail right part		65.30.01.02.02.47
3	1	653004	steel tube incl. riveting		65.30.01.04.00.24
4	1	653005	tube cover - rubber 260 mm		65.30.03.01.00.47
5-6	4	653406	plug for nut M6 incl hexagon nut		65.22.20.10.00.00
6	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
7-10	2	652220	angle adjusting element metal 307° compl with eccentric clamp and mounting set		65.22.20.99.00.10
8	1	652224	eccentric clamp length 35 mm M6x20, black (incl. spring, nut, washer, bush)		65.22.24.99.00.76
9	1		lens head screw with hexagon socket ISO 7380 - M6x25		01.42.08.56.65.00
10	1		spring ring DIN 127- A6		02.85.30.60.05.10
11-13	2	653202	profile tube support 200		65.31.02.01.00.24
12	1	046350200010	clamp conductor black, ø 18-20		04.63.50.20.00.10
13	1	650160	locking unit for push handles		65.01.30.03.00.10

Push bow with synthetic joint up to SW30


At SW20/22 and BH25/30 only limited height adjustment possible

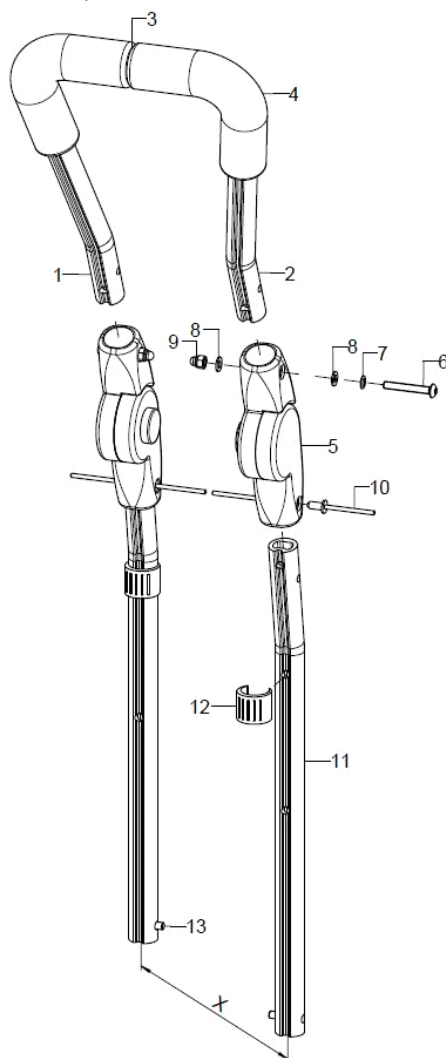

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-13	1	65330101			XX.XX.XX.XX.XX.XX
1-4	1	65330104			XX.XX.XX.XX.XX.XX
1	1	65330105			65.09.02.01.01.77
2	1	65330106			65.09.02.01.02.77
3	1	65330110			65.09.02.02.00.77
4	2	653011			04.65.19.21.00.00
o.B.	2	653005	tube cover - rubber 260 mm		65.30.03.01.00.47
5-10	2	653403	synthetic joint with push knob 205° incl. mounting set		04.03.01.20.01.00
6	1		lens head screw with hexagon socket ISO 7380 - M5x35		01.42.07.99.65.00
7	1		lock washer S5		02.99.10.50.01.10
8	2		washer DIN 125 - A 5,3		02.61.05.31.01.20
9	1		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
10	2		blind rivet F5x10 A2/A2		03.64.00.50.10.30
11-13	2	653202	profile tube support 200		65.31.02.01.00.24
12	1	046350200010	clamp conductor black, ø 18-20		04.63.50.20.00.10
13	1	650160	locking unit for push handles		65.01.30.03.00.10

Push bow with metal joint from SW32


At BH25/30 only limited height adjustment possible

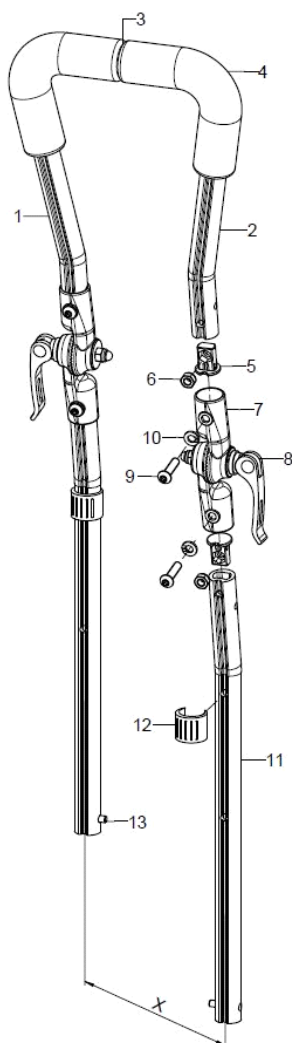

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-13	1	65330102			XX.XX.XX.XX.XX.XX
1-4	1	65330104			XX.XX.XX.XX.XX.XX
1	1	65330105			65.09.02.01.01.77
2	1	65330106			65.09.02.01.02.77
3	1	65330110			65.09.02.02.00.77
4	2	653011			04.65.19.21.00.00
o.B.	2	653005	tube cover - rubber 260 mm		65.30.03.01.00.47
5-6	4	653406	plug for nut M6 incl hexagon nut		65.22.20.10.00.00
6	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
7-10	2	652220	angle adjusting element metal 307° compl with eccentric clamp and mounting set		65.22.20.99.00.10
8	1	652224	eccentric clamp length 35 mm M6x20, black (incl. spring, nut, washer, bush)		65.22.24.99.00.76
9	1		lens head screw with hexagon socket ISO 7380 - M6x25		01.42.08.56.65.00
10	1		spring ring DIN 127- A6		02.85.30.60.05.10
11-13	2	653202	profile tube support 200		65.31.02.01.00.24
12	1	046350200010	clamp conductor black, ø 18-20		04.63.50.20.00.10
13	1	650160	locking unit for push handles		65.01.30.03.00.10

Side truss pad

SORG truss pad system not compatible with truss pad system from Stealth. Applies to deliveries before June 2016.

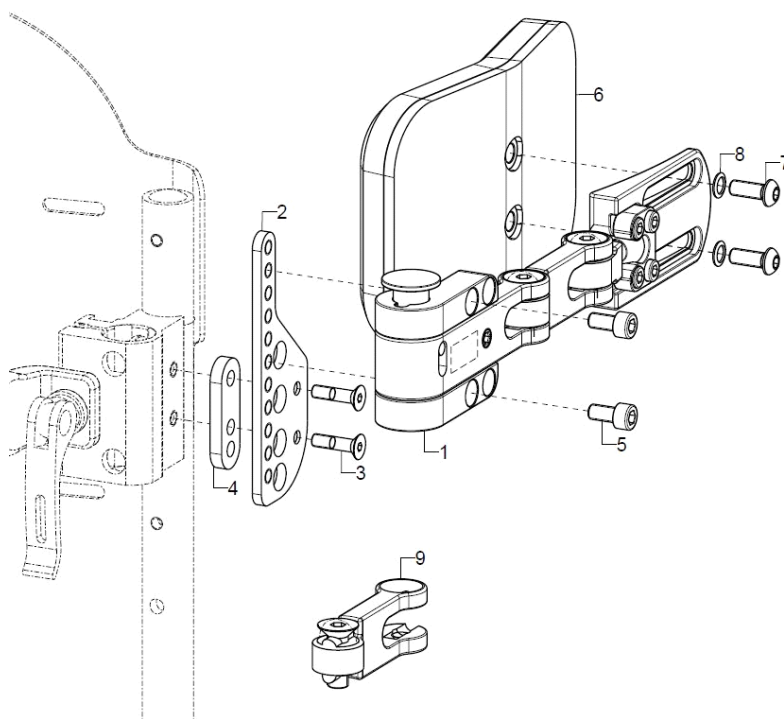


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	68590102			68.59.01.00.02.00
o.B.	1	68590101			68.59.01.00.01.00
altern.	1	68590202			68.59.02.00.02.00
o.B.	1	68590201			68.59.02.00.01.00
2-5	1	68590113			68.59.01.25.00.00
o.B.	1	68590114			68.59.02.25.00.00
3	2		counter sunk screw with hexagon socket DIN 7991 M5x20		01.65.07.94.05.00
4	1				35.33.01.10.00.73
5	2		cylinder screw DIN 912 M5x12		01.22.07.89.72.40
o.B.	2		cylinder screw DIN 912 M6x12		01.22.08.49.62.40
6-8	1	68590104			68.59.01.21.00.00
o.B.	1	68590105			68.59.01.22.00.00
o.B.	1	68590106			68.59.01.26.00.00
o.B.	1	68590204			68.59.01.23.00.00
o.B.	1	68590205			68.59.01.24.00.00
7	2		lens head screw with hexagon socket ISO 7380 - M5x16		01.42.07.92.65.00
o.B.	2		lens head screw with hexagon socket ISO 7380 - M6x16		01.42.08.52.65.00
8	2		lock washer S5		02.99.10.50.01.10
o.B.	2		lock washer S6		02.99.10.60.01.10
9	1	68590103			68.59.01.15.00.00
o.B.	1	68590203			68.59.02.15.00.00




Seitenpelotte Gr. 1 nicht möglich bei:
RH25 und 22" oder 24" Antriebsrad

Seitenpelotte Gr. 2 nicht möglich bei:
RH25 und 22" oder 24" Antriebsrad
RH30 und 24" Antriebsrad


Head rest

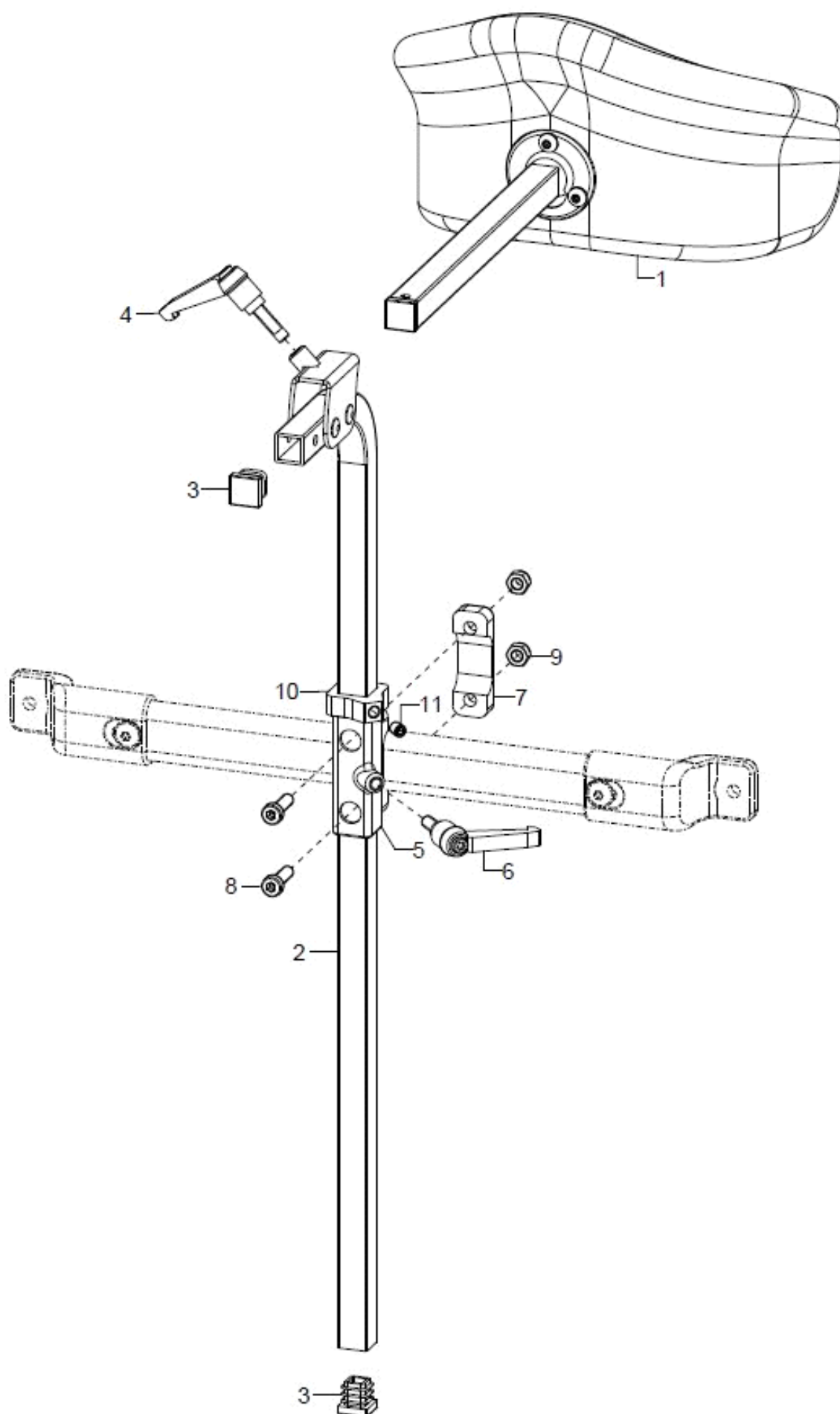

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	686031	head rest cushion model 1, 213x145 mm shell form complete		68.60.15.99.00.10
altern.	1	686033	head rest model 3, 225x100 curved complete		68.60.16.99.00.10
altern.	1	686034	head rest model 4, 300x150 curved complete		68.60.19.99.00.10
o.B.	1	68605001			
2-4	1	68600801			68.60.08.99.01.00
3	2		plugs for square tubes, black 15x15		04.63.11.15.00.10
4	1	403110	clamp lever adjustable M6x20, black		04.13.10.60.20.50
5-9	1	68600810			68.60.01.01.00.00
6	1	45320303			04.13.10.60.16.50
7	1				68.63.01.03.00.00
8	2				01.24.08.53.05.00
9	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
10+11	1				68.60.08.06.00.00
11	1				01.67.07.87.61.21

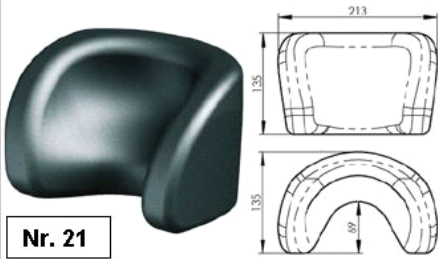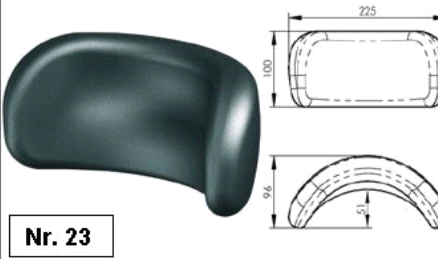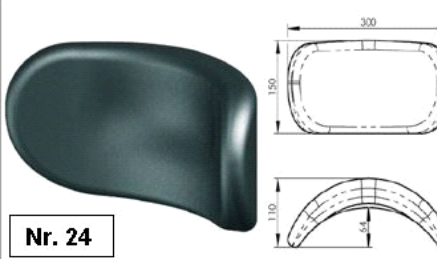Modell 1	Modell 3	Modell 4
 Nr. 21	 Nr. 23	 Nr. 24
muschelförmig	bogenförmig - klein	bogenförmig - groß

Light weight head rest

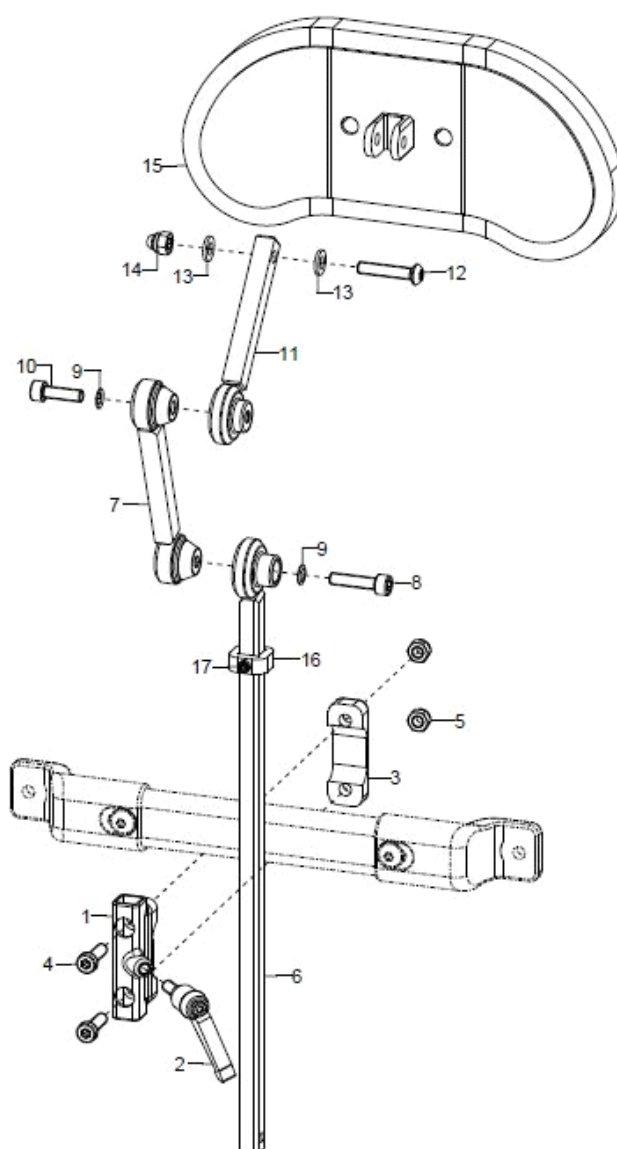


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-17	1	68600920			
o.B.	1	68600921			
1-5	1	68600915			68.63.01.02.00.00
2	1		clamp lever adjustable M6x16, black		04.13.10.60.16.50
3	1				68.63.01.03.00.00
4	2				01.24.08.53.05.00
5	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
6-14	1	68600918			
6	1	68600901			68.60.09.01.00.00
7	1	68600903			68.60.09.03.00.00
8	1		cylinder screw DIN 912 M6x25		01.22.08.56.02.40
9	2		lock washer S6		02.99.10.60.01.10
10	1				01.22.08.54.05.00
11	1	68600904			68.60.09.04.00.00
12	1		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
13	2		washer DIN 125 - A 6,4		02.61.06.41.41.20
14	1		hexagon cap nut low form with clamp DIN 986 - M6		02.33.00.60.11.20
12-15	1	68600910			




o.B.	1	68600911	
16	1		68.60.09.09.00.00
17	1		01.67.07.87.61.21

Connection for back shell

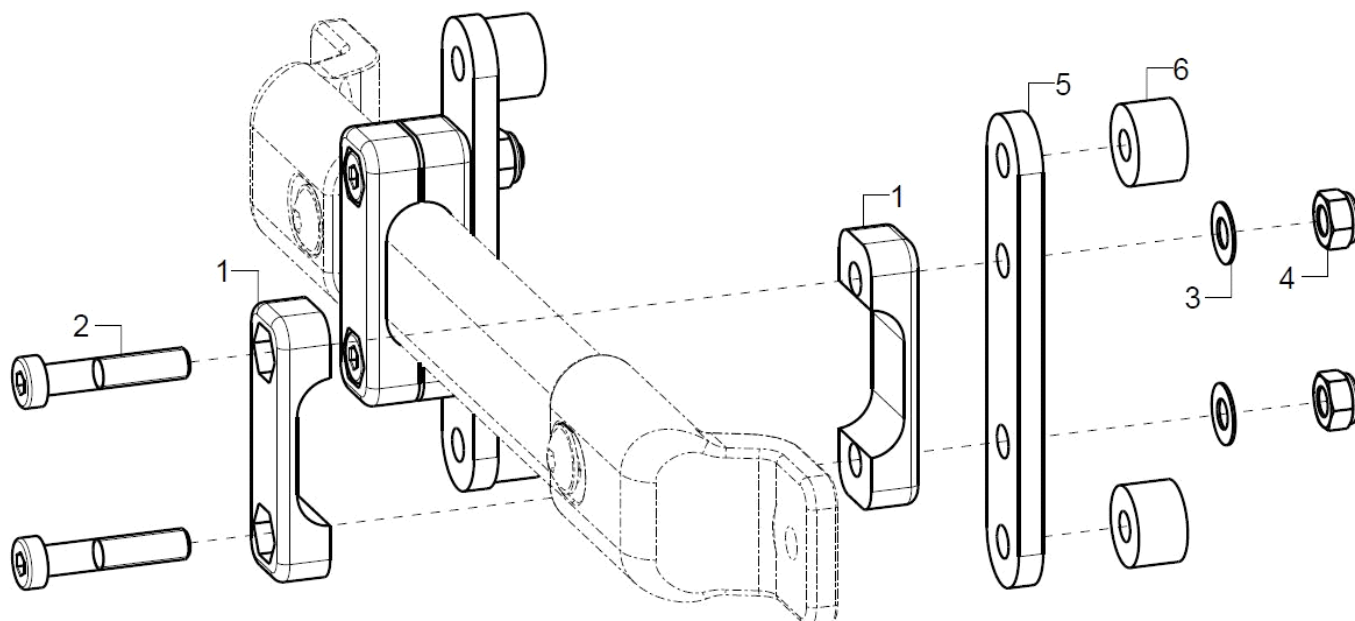

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-6	1/2				
1	2				68.63.01.03.00.00
2	2				01.24.08.58.05.00
3	2		fitted washer DIN 988 - 6x12x0,5 - A2		02.68.06.00.50.30
4	2		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
5	1				69.80.04.01.00.80
6	2	35310209			45.31.01.03.00.72

Side part

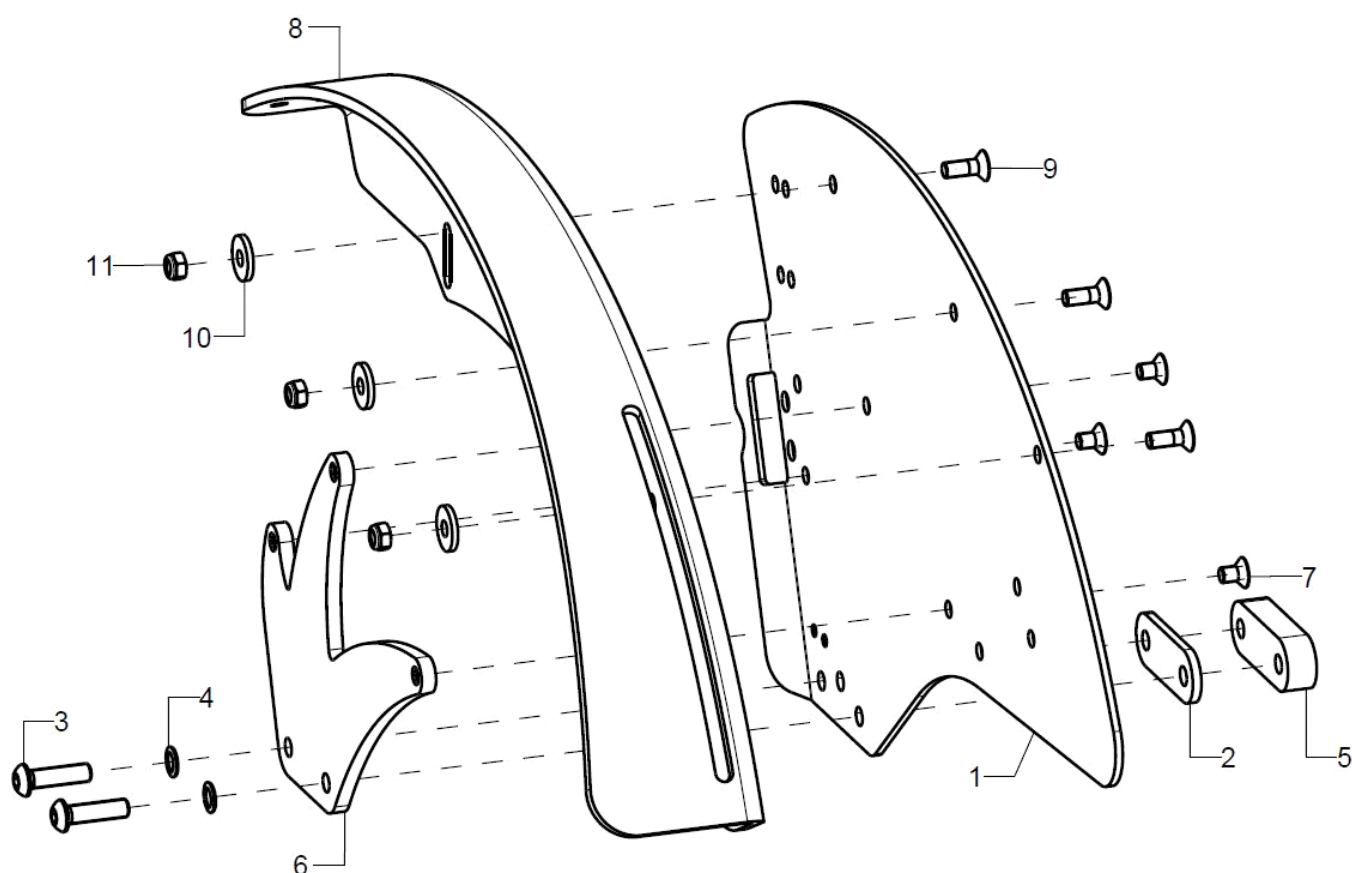

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-5	1	45032002			45.03.20.02.20.77
altern.	1	45032202			45.03.20.02.22.77
altern.	1	45032402			45.03.20.02.24.77
o.B.	1	45032001			45.03.20.01.20.77
altern.	1	45032201			45.03.20.01.22.77
altern.	1	45032401			45.03.20.01.24.77
2-8	2	450398			
2	1				45.03.20.05.00.77
3	2				01.42.08.55.65.00
o.B.	2		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
4	2		lock washer S6		02.99.10.60.01.10
5+3	2	45031200			45.03.12.01.00.78
6+7	2	45032003			45.03.20.02.00.77
7	3		counter sunk screw with hexogon socket DIN 7991 M5x8		01.65.07.87.05.00
8	1	50220102	wheel cover 20" right		50.22.01.01.02.72
altern.	1	50220202	wheel cover 22" right		50.22.01.02.02.72




Spare parts list Mio Move

Side part

Page 44 of 79

altern.	1	50220302	wheel cover 24" right	50.22.01.03.02.72
o.B.	1	50220101	wheel cover 20" left	50.22.01.01.01.72
altern.	1	50220201	wheel cover 22" left	50.22.01.02.01.72
altern.	1	50220301	wheel cover 24" left	50.22.01.03.01.72
9-11	1	50220199	mounting set wheel cover	XXXXXXXXXXXX
9	3		counter sunk screw with hexagon socket DIN 7991 M5x14	01.65.07.90.05.00
10	3		washer DIN 7349 - 5,3	02.77.05.22.00.20
11	3		hexagon nut self-locking low form DIN 985 - M5	02.32.00.50.01.20

Side part wheel base extension

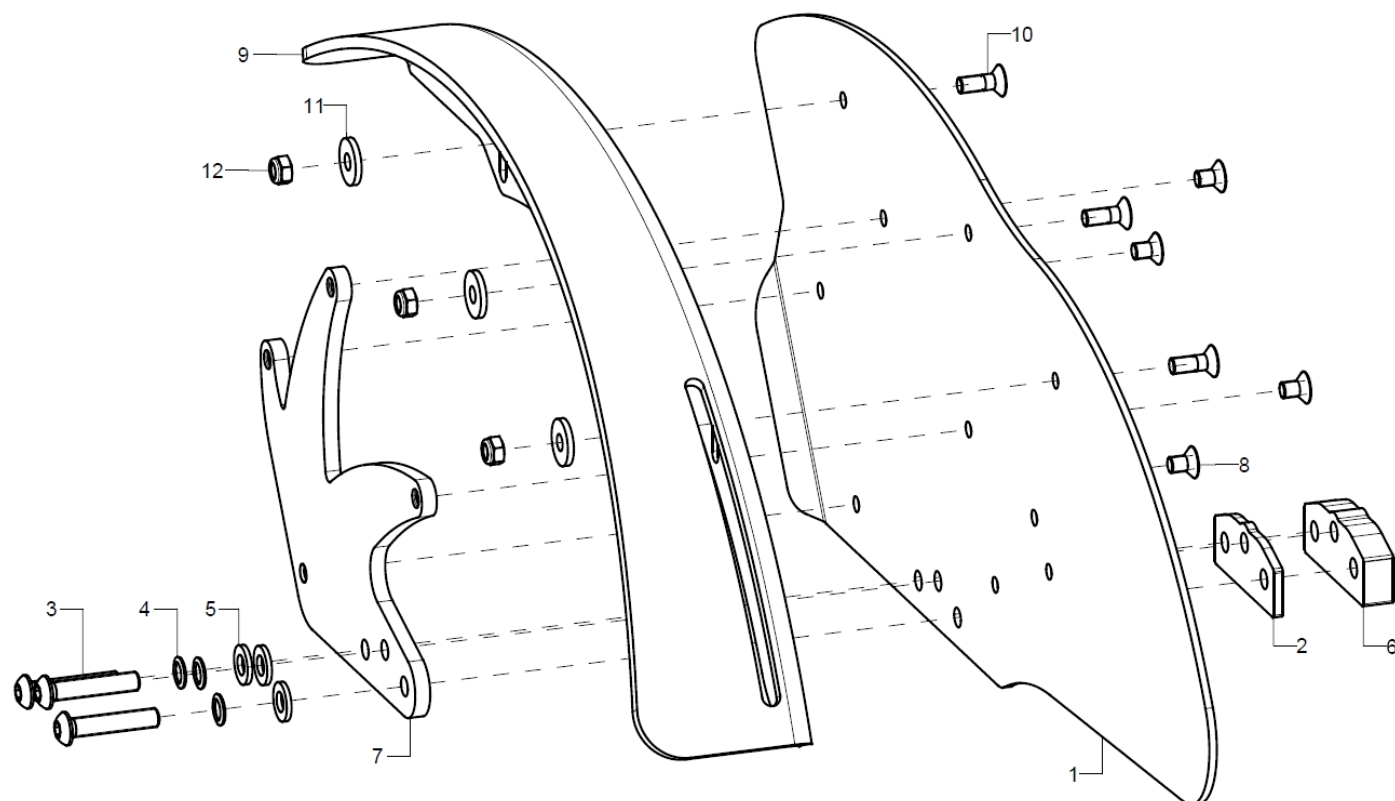

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-5	1	45031302			45.03.13.01.00.78
altern.	1	45031402			45.03.14.01.02.78
altern.	1	45031502			45.03.15.01.02.78
o.B.	1	45031301			45.03.13.01.01.78
altern.	1	45031401			45.03.14.01.01.78
altern.	1	45031501			45.03.15.01.01.78
2-8	2	450397			
2	1				45.03.20.05.00.77
3	3		lens head screw with hexagon socket ISO 7380 - M6x30		01.42.08.58.65.00
o.B.	3		lens head screw with hexagon socket ISO 7380 - M6x40		01.42.08.60.65.00
4	3		lock washer S6		02.99.10.60.01.10
5	3		washer DIN 125 - A 6,4		02.61.06.41.41.20
6+3	1	45031200			45.03.12.01.00.78
7+8	1	45032004			45.03.18.01.00.78
8	4		counter sunk screw with hexagon socket DIN 7991 M5x8		01.65.07.87.05.00
9	1	50220102	wheel cover 20" right		50.22.01.01.02.72




Spare parts list Mio Move

Side part

Page 46 of 79

altern.	1	50220101	wheel cover 20" left	50.22.01.01.01.72
altern.	1	50220202	wheel cover 22" right	50.22.01.02.02.72
o.B.	1	50220201	wheel cover 22" left	50.22.01.02.01.72
altern.	1	50220302	wheel cover 24" right	50.22.01.03.02.72
altern.	1	50220301	wheel cover 24" left	50.22.01.03.01.72
10-12	1	50220199	mounting set wheel cover	XXXXXXXXXXXX
10	3		counter sunk screw with hexagon socket DIN 7991 M5x14	01.65.07.90.05.00
11	3		washer DIN 7349 - 5,3	02.77.05.22.00.20
12	3		hexagon nut self-locking low form DIN 985 - M5	02.32.00.50.01.20

Arm pad for side guards

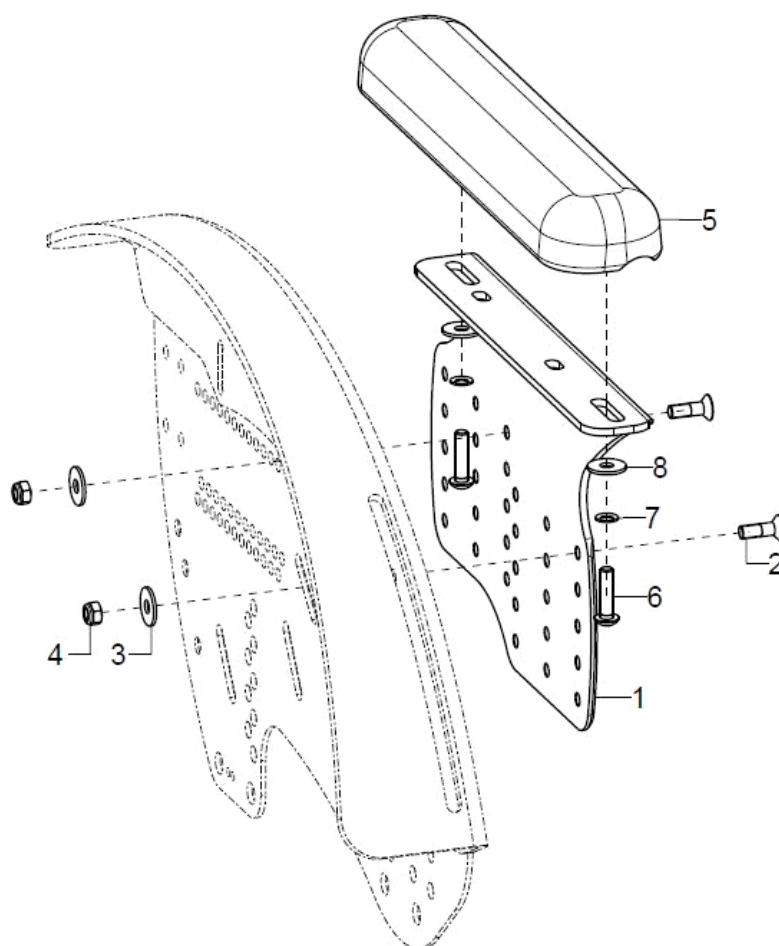

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-8	1P	45220701	arm rest holder		
o.B.	1P	45220702	holder arm rest		
1-4	1P	45220705			45.03.20.20.00.77
o.B.	1P	45220706			45.03.20.21.00.77
2	2		counter sunk screw with hexagon socket DIN 7991 M5x16		01.65.07.92.05.00
3	2		washer DIN 9021 - M5		02.79.05.31.20.20
4	2		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
5-8	2	450009			04.65.01.17.00.10
o.B.	2	450007	arm rest, straight black 240x50x26, 2xM5x164		04.65.01.24.00.10
6	2		lens head screw with hexagon socket ISO 7380 - M5x16		01.42.07.92.65.00
o.B.	2				01.42.07.64.65.00
7	2		lock washer S5		02.99.10.50.01.10
8	2		washer DIN 9021 - M5		02.79.05.31.20.20




Caster wheels


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
o.B.	2	2812	caster 4" , PU		04.52.42.04.00.30
o.B.	2	2804	steering wheel 4 solid rubber		04.52.41.04.00.20
o.B.	2	2820	caster 4" PU with LED		04.52.45.04.00.10
o.B.	2	2835	caster 4" solid rubber black with aluminum rim		04.52.41.04.00.31
o.B.	2	2805	caster 5" PU		04.52.42.05.00.30
o.B.	2	2809	caster 5" soft solid rubber		04.52.44.05.00.10
o.B.	2	2834	caster 5" PU, transparent with LED		04.52.45.05.00.40
o.B.	2	2836	caster 5" solid rubber black with aluminum rim		04.52.41.05.00.61
o.B.	2	289998			

Caster wheel fork

Stier-Rahmen

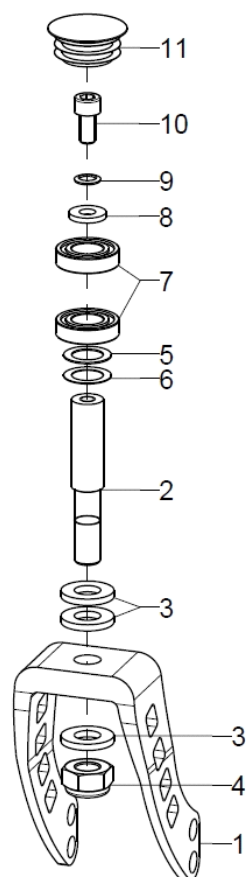


Retro-Rahmen

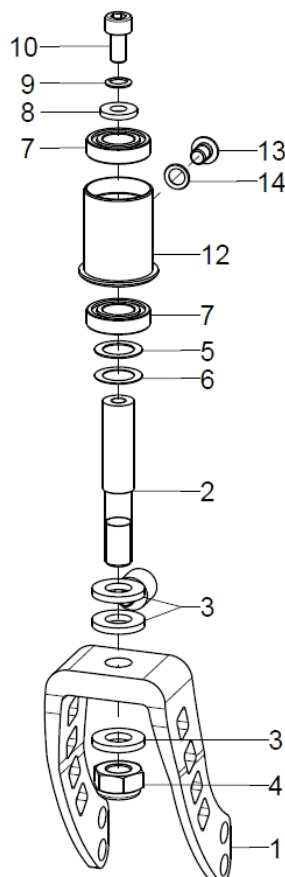

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	2	2843	caster fork aluminum 100x45		28.43.01.00.00.10
2	2	282104	steering axle $\varnothing 12 \times 65$, 1		28.21.05.01.00.10
o.B.	2	285001	steering axle 65 for locking		28.50.00.01.00.10
o.B.	2	285101	steering axle 65 for track fixation		28.50.00.03.00.10
3-4	2	282120	mounting set caster axle on caster fork aluminum (washers, hexagon nut)		28.21.20.01.01.00
3	3		washer DIN 1440 - 10x20x2,5		02.71.10.02.50.20
4	1		hexagon nut self-locking low form DIN 985 - M10		02.32.01.00.01.20
5-11	2	282127			28.21.27.99.00.73
5	1		fitted washer DIN 988 - 12x18x0,5		02.68.12.00.50.30
6	1		fitted washer DIN 988 - 12x18x0,3		02.68.12.00.30.30
7	1		grooved ball bearing $\varnothing 24/12 \times 6$		04.32.12.06.00.10
8	1		washer reinforced $\varnothing 14/6,2 \times 2$		58.30.15.01.00.24
9	1		lock washer S6		02.99.10.60.01.10
10	1		cylinder screw DIN 912 M6x12		01.22.08.49.62.40
11	1	282129			04.63.10.30.00.11
5-10+12-14	2	282125	mounting set caster axle for Mio Retro		28.21.25.99.00.73
12	1				28.00.73.01.00.73
13	1		lens head screw with hexagon socket ISO 7380 - M6x8		01.42.08.47.65.00
14	1		lock washer S6		02.99.10.60.01.10

Track locking


Only in conjunction with steering wheel axle for track fixation.

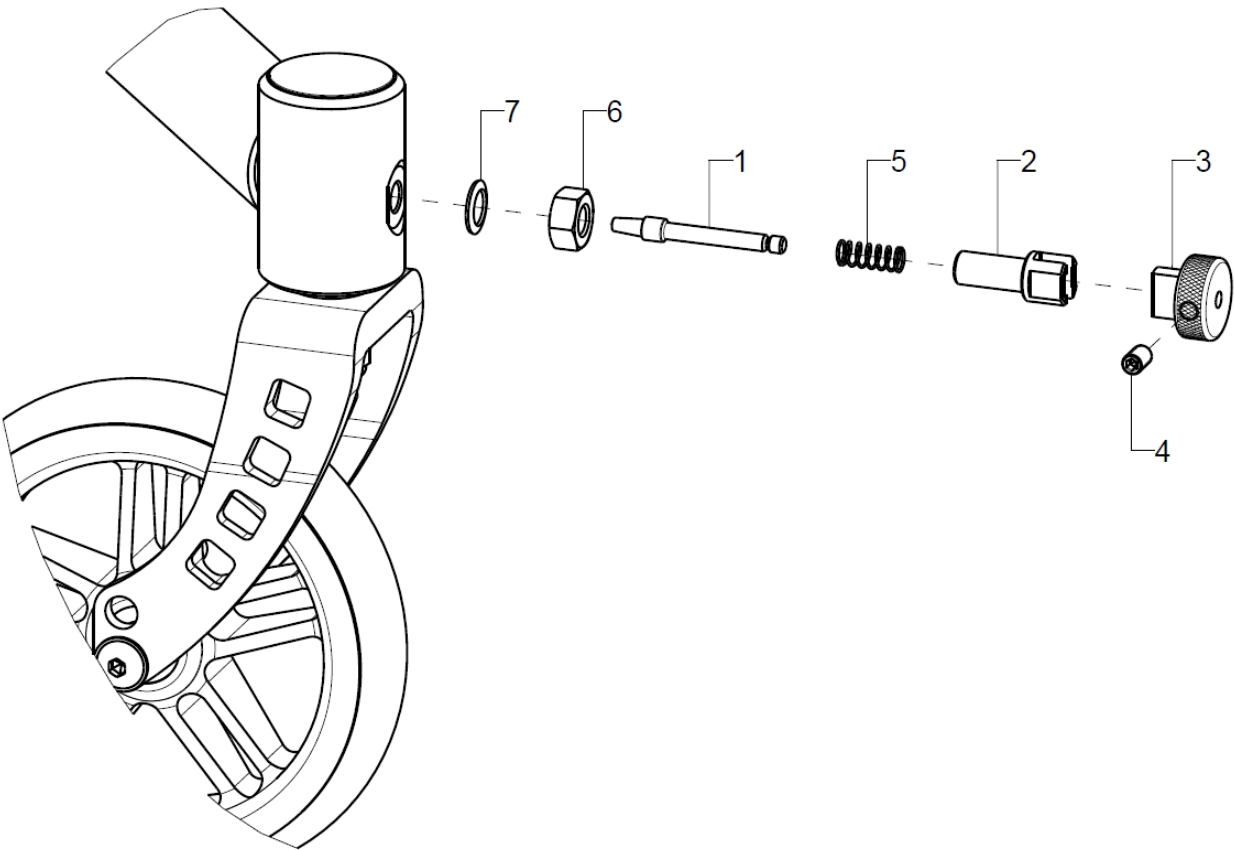

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-7	2	285003	track fixation complete		XXXXXXXXXXXX
1	1				28.50.01.01.00.10
2	1				28.50.01.02.00.10
3	1				28.50.01.03.00.10
4	1		setscrew with hexagon socket DIN 913,M5x8		01.67.07.87.61.20
5	1		pressure spring 0,55x5,5x23,7		04.21.05.52.37.10
6	1		hexagon nut DIN 934 - M8		02.19.00.80.01.20
7	1		lock washer S8		02.99.10.80.01.10


Track fixation

Only in conjunction with steering wheel axle for track fixation.

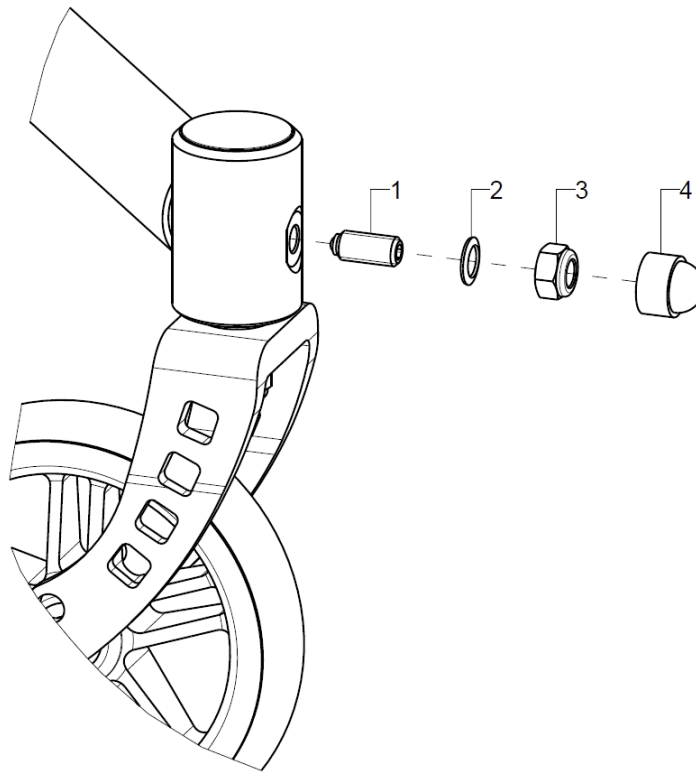

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-4	2	285103	spring pressure piece M8 complete		04.14.50.80.21.20
2	1		lock washer S8		02.99.10.80.01.10
3	1		hexagon nut self-locking low form DIN 985 - M8		02.32.00.80.01.20
4	1		hexagon protection cap for SW13, black (M8)		04.63.06.13.00.10


Axle adapter

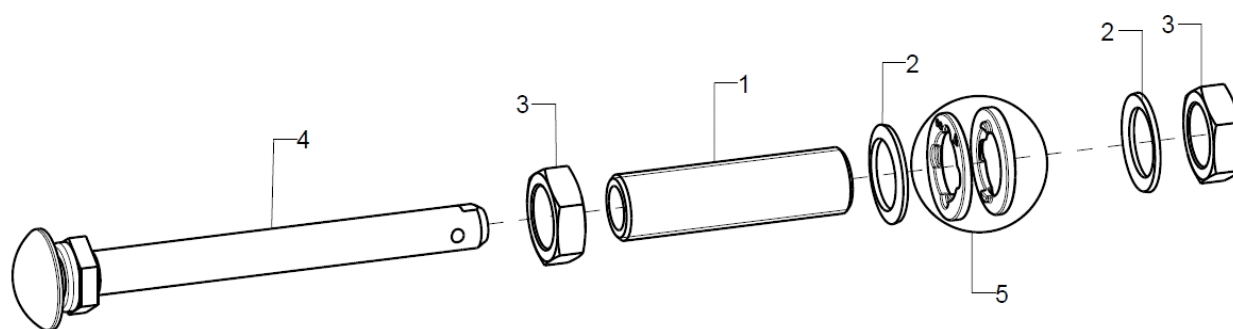

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-3	1	210026	axle adapter M18x1 d12,7 65mm incl. mounting set		04.51.61.26.52.02
2	2		lock washer S18		02.99.11.80.01.10
3	2		hexagon nut M18x1		02.19.01.30.11.21
4	2	210003	quick-release axle 12,7x103 mm for standard hub from 5/2013 and light weight wheel		04.51.51.21.00.31
5	2	20150503			20.15.05.03.01.00
o.B.	2	20150501			20.15.05.01.01.00
o.B.	2	20150504			


Drum brake anchor

Only possible with 11° wheel camber.

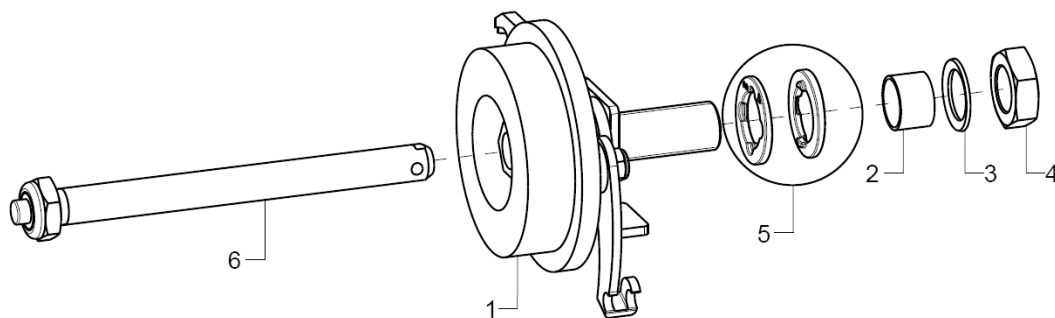

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	2	220005	drum brake body complete, incl. adapter and counter wrench		22.00.01.99.00.10
o.B.	2	220013	drum brake anchor pre-assembled		
2	2				22.00.12.02.00.69
3	2		lock washer S16		02.99.11.60.01.10
4	2		nut M16x1 SW 24		02.19.01.60.11.21
5	2	20150503			20.15.05.03.01.00
o.B.	2	20150504			
6	1	220001	quick-release axle D12x123mm for drum brake		04.51.51.21.20.30

Wheel base extension

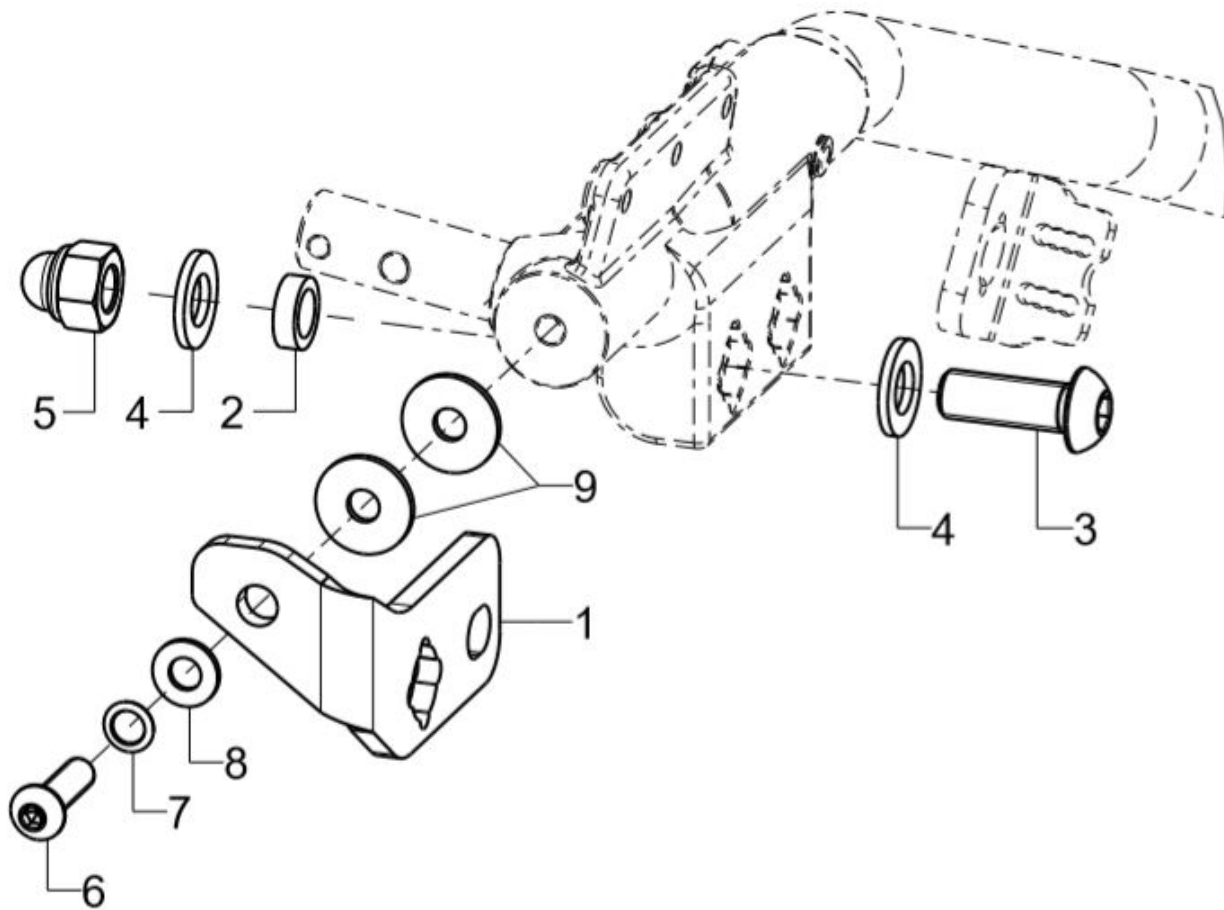


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-8	1	767802			76.01.77.01.02.77
o.B.	1	767801			76.01.77.01.01.77
2	1	767803			76.78.01.03.00.78
3	1				
4	2				02.61.13.02.31.20
5	1				
6	1				01.42.09.78.65.00
7	1		lock washer S8		02.99.10.80.01.10
8	1		washer DIN 125 - A 8,4		02.61.08.41.41.20
9	2				02.79.08.41.60.21



Rear wheels

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
o.B.	2	212000	driving wheel 20" with standard hub		
o.B.	2	212200	22" driving wheel with standard hub		
o.B.	2	212400	24" driving wheel standard hub		
o.B.	2	222000	20" driving wheel with drum brake		
o.B.	2	222200	22" driving wheel with drum brake		
o.B.	2	222400	24" driving wheel standard with drum brake		
o.B.	2	212040	20" profile wheel with built-in hand rim, standard hub		
o.B.	2	212240	22" profile wheel with built-in hand rim, standard hub		
o.B.	2	212440	24" profile wheel with built-in hand rim, standard hub		
o.B.	2	222040	20" profile wheel with built-in hand rim with drum brake		
o.B.	2	222240	22" profile wheel with built-in hand rim with drum brake		
o.B.	2	222440	24" profile wheel with built-in hand rim with drum brake		
o.B.	2	212077	20" lightweight wheel with hand rim, standard		
o.B.	2	212277	22" lightweight wheel with hand rim, standard		
o.B.	2	212477	24" lightweight wheel with hand rim, standard		
o.B.	2	212081	20" lightweight wheel with drum brake hub, black		
o.B.	2	212281	22" lightweight wheel with drum brake, black		
o.B.	2	212481	24" lightweight wheel with drum brake, black		
o.B.	2	200011	groove ball bearing ø28/12 6001z		04.32.12.08.00.10


Double handrim Standard

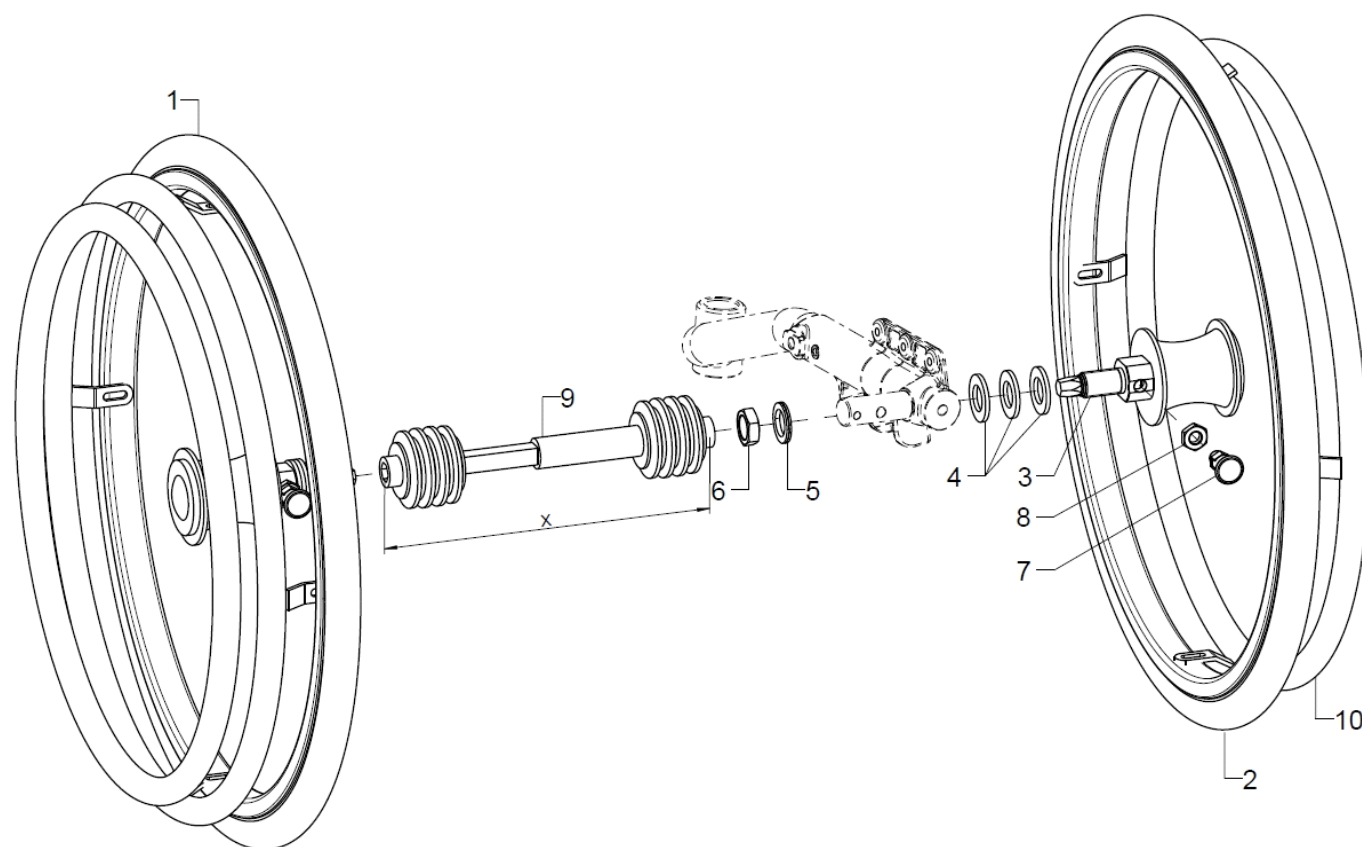

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-9	1	261401	double handrim 20" standard hub incl. drag anchor, brake arm and telescope rod		XXXXXXXXXX
1-9	1	261403	double handrim 22" standard hub incl. drag anchor, brake arm and telescope rod		XXXXXXXXXX
1-9	1	261405	double handrim 24" standard hub incl. drag anchor, brake arm and telescope rod		XXXXXXXXXX
1+3-9	1	261440	20" double handrim driving wheel standard hub, driving side		XXXXXXXXXX
1+3-9	1	261442	22" double handrim driving wheel standard hub, driving side		XXXXXXXXXX
1+3-9	1	261444	24" double handrim driving wheel standard hub, driving side		XXXXXXXXXX
2-8	1	261441	20" double handrim driving wheel standard hub, opposite side		XXXXXXXXXX
2-8	1	261443	22" double handrim driving wheel standard hub, opposite side		XXXXXXXXXX
2-8	1	261445	24" double handrim driving wheel standard hub, opposite side		XXXXXXXXXX
3-8	2	261429	mounting set DHR drum brake 3-8		XXXXXXXXXX
3	1				
4	3		washer DIN 125 - B 19		02.61.19.02.72.20
5	1		lock washer M18 (Nord-Lock)		02.99.21.95.01.10
6	1		hexagon nut M18x1		02.19.01.30.11.21
7	1				04.14.34.50.01.00
8	1		hexagon nut DIN 934 - M10		02.19.01.00.01.20
9	1	26140002			26.14.10.01.02.00
o.B.	1	26140003			26.14.10.01.03.00
o.B.	1	26140004			26.14.10.01.04.00
o.B.		26140005			26.14.10.01.05.00
o.B.	1	26140006			26.14.10.01.06.00

		Spare parts list Mio Move		Rear wheel	Page 57 of 79
10	1	261430	hand rim for other side 20"		XXXXXXXXXXXX
o.B.	1	261431	hand rim for other side 22"		XXXXXXXXXXXX
o.B.	1	261432	hand rim for other side 24"		XXXXXXXXXXXX

Double handrim drum brake

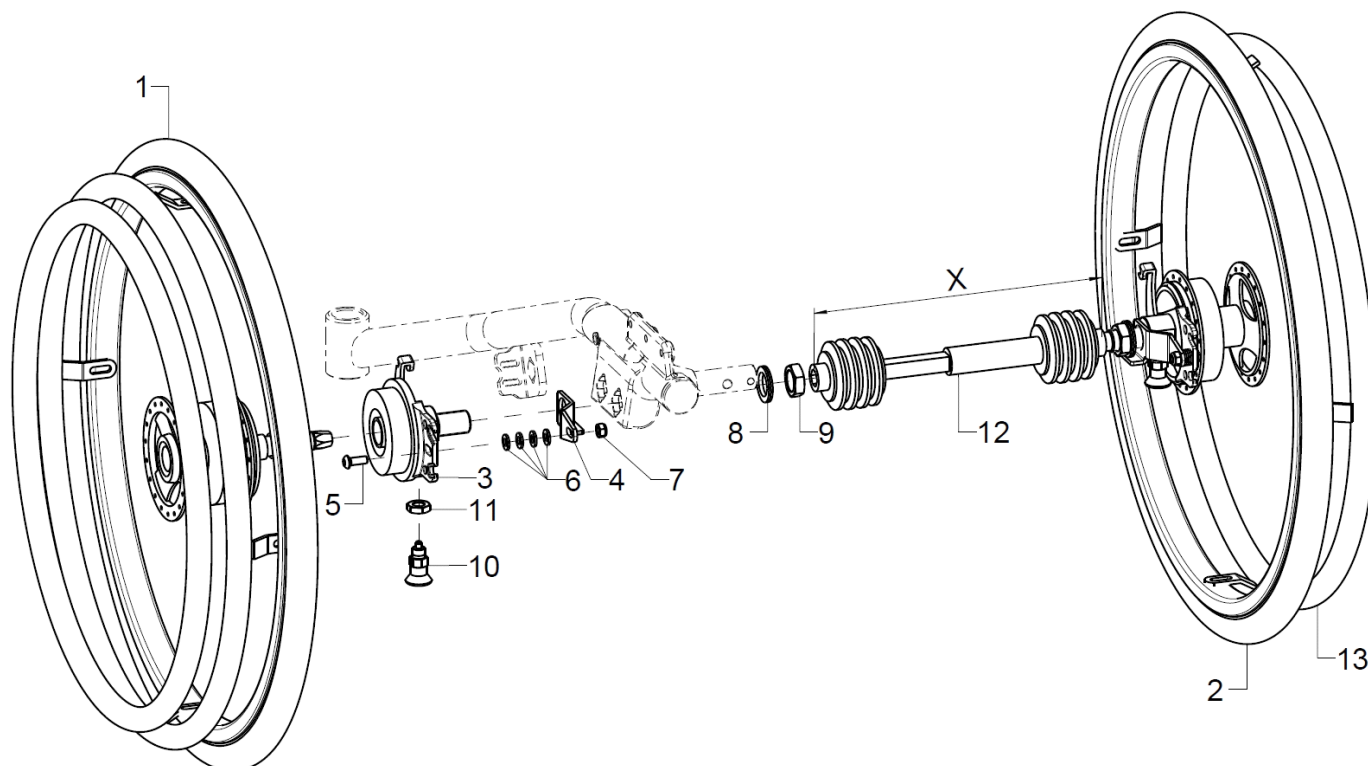

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-12	1	261402	double handrim 20" drum brake hub incl. drag anchor, brake arm and telescope rod		XXXXXXXXXX
o.B.	1	261404	double handrim 22" drum brake hub incl. drag anchor, brake arm and telescope rod		XXXXXXXXXX
o.B.	1	261406	double handrim 24" drum brake hub incl. drag anchor, brake arm and telescope rod		XXXXXXXXXX
1	1	261450	20" double handrim driving wheel drum brake hub, driving side		XXXXXXXXXX
o.B.	1	261452	22" double handrim driving wheel drum brake hub, driving side		XXXXXXXXXX
o.B.	1	261454	24" double handrim driving wheel drum brake hub, driving side		XXXXXXXXXX
2	1	261451	20" double handrim driving wheel drum brake hub, opposite side		XXXXXXXXXX
o.B.	1	261453	22" double handrim driving wheel drum brake hub, opposite side		XXXXXXXXXX
o.B.	1	261455	24" double handrim driving wheel drum brake hub, opposite side		XXXXXXXXXX
3-11	1	261428	mounting set DHR drum brake		XXXXXXXXXX
3	1				
4	1				22.00.78.02.00.78
5	1		lens head screw with hexagon socket ISO 7380 - M6x20		01.42.08.54.05.00
6	4		washer DIN 125 - A 6,4		02.61.06.41.41.20
7	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
8	1		lock washer M18 (Nord-Lock)		02.99.21.95.01.10
9	1		hexagon nut M18x1		02.19.01.30.11.21
10	1				04.14.34.50.01.00
11	1		hexagon nut DIN 934 - M10		02.19.01.00.01.20
12	1	26140002			26.14.10.01.02.00
o.B.	1	26140003			26.14.10.01.03.00



o.B.	1	26140004		26.14.10.01.04.00
o.B.	1	26140005		26.14.10.01.05.00
o.B.	1	26140006		26.14.10.01.06.00
13	1	261430	hand rim for other side 20"	XXXXXXXXXXXX
o.B.	1	261431	hand rim for other side 22"	XXXXXXXXXXXX
o.B.	1	261432	hand rim for other side 24"	XXXXXXXXXXXX



Tyres

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
o.B.	2	202016			
o.B.	2	202216			
o.B.	2	202416			
o.B.	2	202212	tire for pneumatic tube 22x1" Marathon plus, black		
o.B.	2	202412	tire for pneumatic tube 24x1" Marathon plus, black		04.61.12.41.20.21
o.B.	2	200520	tube 20"		
o.B.	2	200522	tube 22"		
o.B.	2	200524	tube 24"		
o.B.	2	232013			
o.B.	2	232213			
o.B.	2	232413			
o.B.	2	232012			04.61.22.00.10.11
o.B.	2	232212			04.61.22.20.10.11
o.B.	2	232412			04.61.22.40.10.11




Handrim

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
o.B.	2	302101	handrim 20" high, stainless steel polished		
o.B.	2	302201	handrim 22" standard, stainless steel polished		
o.B.	2	302301	handrim 22" high, stainless steel polished		
o.B.	2	302401	handrim 24" standard, stainless steel polished		
o.B.	2	302501	handrim 24" high, stainless steel polished		
o.B.	2	3420	Maxgrepp-handrim 20", 4 loops complete		
o.B.	2	3422	Maxgrepp-handrim 22", 4 loops complete		
o.B.	2	3424	Maxgrepp-handrim 24", 4 loops complete		
o.B.	2	3320	hand rim aluminium, 20", black, pvc coating		
o.B.	2	3322	hand rim aluminium, 22", black, pvc coating		
o.B.	2	3324	hand rim aluminium, 24", black, pvc coating		
o.B.	2	342424			
o.B.	2	302078			
o.B.	2	302278			
o.B.	2	302478			

Handrim covers

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
o.B.	2	322002	handrim cover silicone black ø442		04.67.06.20.00.40
o.B.	2	322202	handrim cover silicone black ø478		04.67.06.22.00.30
o.B.	2	322205	handrim cover silicone black ø453		04.67.06.22.00.20
o.B.	2	322208	handrim cover silicone black ø388		04.67.06.22.00.10
o.B.	2	322402	handrim cover silicone black ø532		04.67.06.24.00.30
o.B.	2	322404	handrim cover silicone black ø504		04.67.06.24.00.10
o.B.	2	322405	handrim cover silicone black ø514		04.67.06.24.00.20
o.B.	2	322603	handrim cover silicone black ø565		04.67.06.24.00.40
o.B.	2	322001	handrim cover profile wheel, Ergo 20"		
o.B.	2	322201	handrim cover profile wheel, Ergo 22"		
o.B.	2	322401	handrim cover profile wheel, Ergo 24"		

Spoke cover

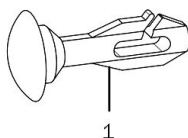


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	250001	clip for transparent spoke cover		25.00.99.01.00.10
o.B.	2	252000	spoke cover 20", Unicolor transparent		
o.B.	2	252200	spoke cover 22", Unicolor transparent		
o.B.	2	252400	spoke cover 24", Unicolor transparent		

Design-Dekor-Speichenschützer können Sie unter den folgenden Internetadressen wählen:


www.speichen-schutz.de
www.wheelchairgadgets.com

Bowden cable brake pressure piece

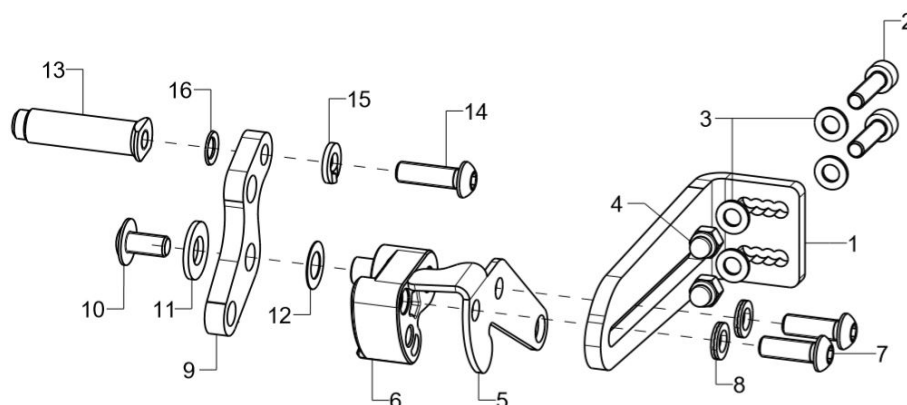


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-4	1	35351004			35.35.10.01.02.78
o.B.	1	35351003			35.35.10.01.01.78
2	2		cylinder screw DIN 912 M5x20		01.22.07.94.72.40
3	4		washer DIN 125 - A 5,3		02.61.05.31.01.20
4	2		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
5-12	1	35360110			35.37.05.00.02.83
o.B.	1	35360109			35.37.05.00.01.83
5	1				35.34.01.01.00.00
6-8	1				35.35.01.01.02.78
o.B.	1				35.35.01.01.01.78
7	2		lens head screw with hexagon socket ISO 7380 - M6x18		01.42.08.53.05.00
o.B.	2		hexagon screw DIN 933 - M6x16		01.04.08.52.02.40
8	2		lock washer M6 (Nord-Lock)		02.99.20.65.01.10
9-12	1				35.34.01.05.00.00
10	1				01.43.08.48.62.61
11	1		washer DIN 125 - A 8,4 - PA black		02.61.08.41.61.71
12	1				02.68.08.03.00.30
13-16	1	352517			35.25.01.21.00.00
o.B.	1	352511			35.25.01.14.00.00
altern.	1	352518	brake bolt 40 mm left-hand thread incl. mounting set		35.25.01.18.00.10
o.B.	1	352506	brake bolt milled 40 mm incl. mounting set		35.25.01.15.00.10
altern.	1	352519	brake bolt 50 mm left-hand thread incl. mounting set		35.25.01.19.00.10
o.B.	1	352507	brake bolt milled 50 mm incl. mounting set		35.25.01.16.00.10
altern.	1	352520			35.25.01.20.00.10
o.B.	1	352510	bracket 34, Bd 16x3x50 lockable		35.25.01.17.00.10
14	1		lens head screw with hexagon socket ISO 7380 , M6x20 left-hand thread		01.42.08.54.05.01
o.B.	1		lens head screw with hexagon socket ISO 7380 - M6x20		01.42.08.54.05.00
15	1		spring ring DIN 127- A6		02.85.30.60.05.10
16	1		lock washer S6		02.99.10.60.01.10

Bowden cable brake control/non-control sides

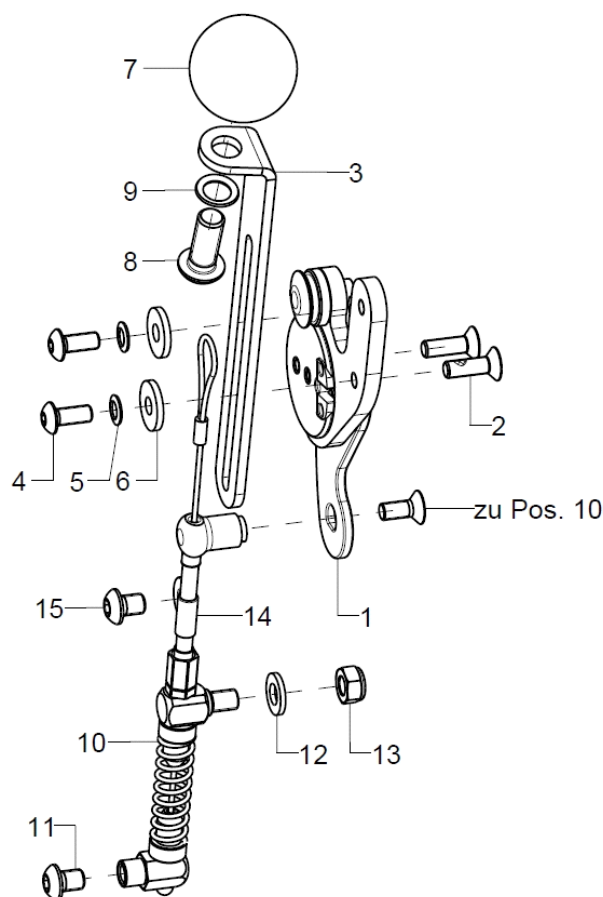


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-2	1	35310504	operating unit for Bowden cable brake right, premounted incl. screws		35.36.01.99.04.78
altern.	1	35310503	operating unit bowden cable brake left		35.36.01.99.03.78
2	2				01.66.02.50.92.20
3	1	35300104	brake lever, 3x20,6x21x127, right		35.30.01.01.02.66
o.B.	1	35300103	brake lever, 3x20,6x21x127, left		35.30.01.01.01.66
4-9	1	35310599	mounting set brake lever extension		XXXXXXXXXXXX
4	2		lens head screw with hexagon socket ISO 7380 - M5x12		01.42.07.89.65.00
5	2		lock washer S5		02.99.10.50.01.10
6	2		washer DIN 7349 - 5,3		02.77.05.22.00.20
7	1		ball knob $\varnothing 32/M8$, form C, black		04.06.13.20.80.50
8	1		lens head screw with hexagon socket ISO 7380 - M8x20		01.42.09.74.65.00
9	1		lock washer S8		02.99.10.80.01.10
10-15	1	35360104			35.36.01.01.00.78
11	1		lens head screw with hexagon socket ISO 7380 - M6x8		01.42.08.47.65.00
12	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
13	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
14+15	1				04.14.20.40.19.01
15	1		lens head screw with hexagon socket ISO 7380 - M6x8		01.42.08.47.65.00

Bowden cable lock

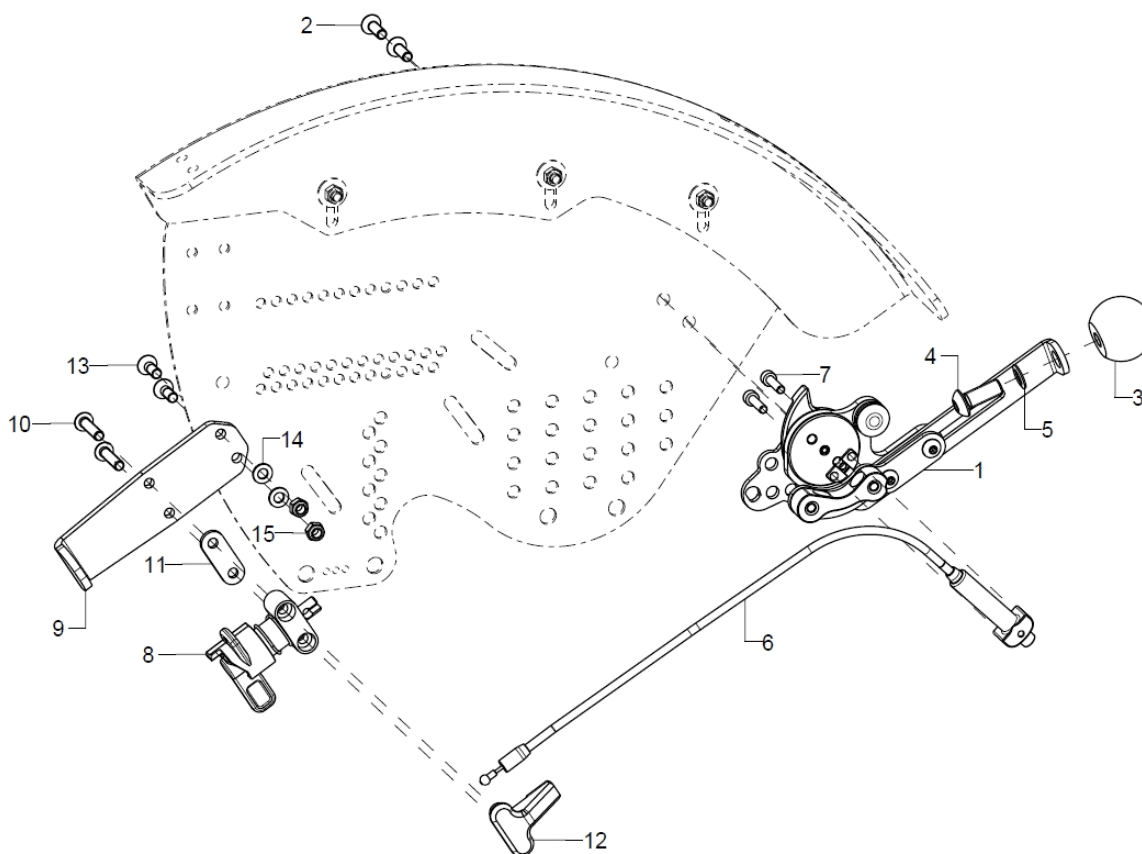


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-15	1	35360302			
o.B.	1	35360301			
1-6	1	35360311			
o.B.	1	35360310			
2	2				01.66.02.50.92.20
3	1		ball knob $\varnothing 32/M8$, form C, black		04.06.13.20.80.50
4	1		lens head screw with hexagon socket ISO 7380 - M8x20		01.42.09.74.65.00
5	1		lock washer S8		02.99.10.80.01.10
6+7	1	35360307			35.36.03.20.00.78
7	2				01.22.07.32.62.40
8-15	1	35360306			
o.B.	1	35360305			
8	1				35.36.03.10.02.78
o.B.	1				35.36.03.10.01.78
9	1				35.36.03.05.00.78
10	2				01.42.07.95.65.00
11	1				35.36.02.07.00.78
12	1				35.36.03.12.00.78
13	2		counter sunk screw with hexagon socket DIN 7991 M5x14		01.65.07.90.05.00
14	2		washer DIN 125 - A 5,3		02.61.05.31.01.20
15	2		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20

Brake for wheel base extension

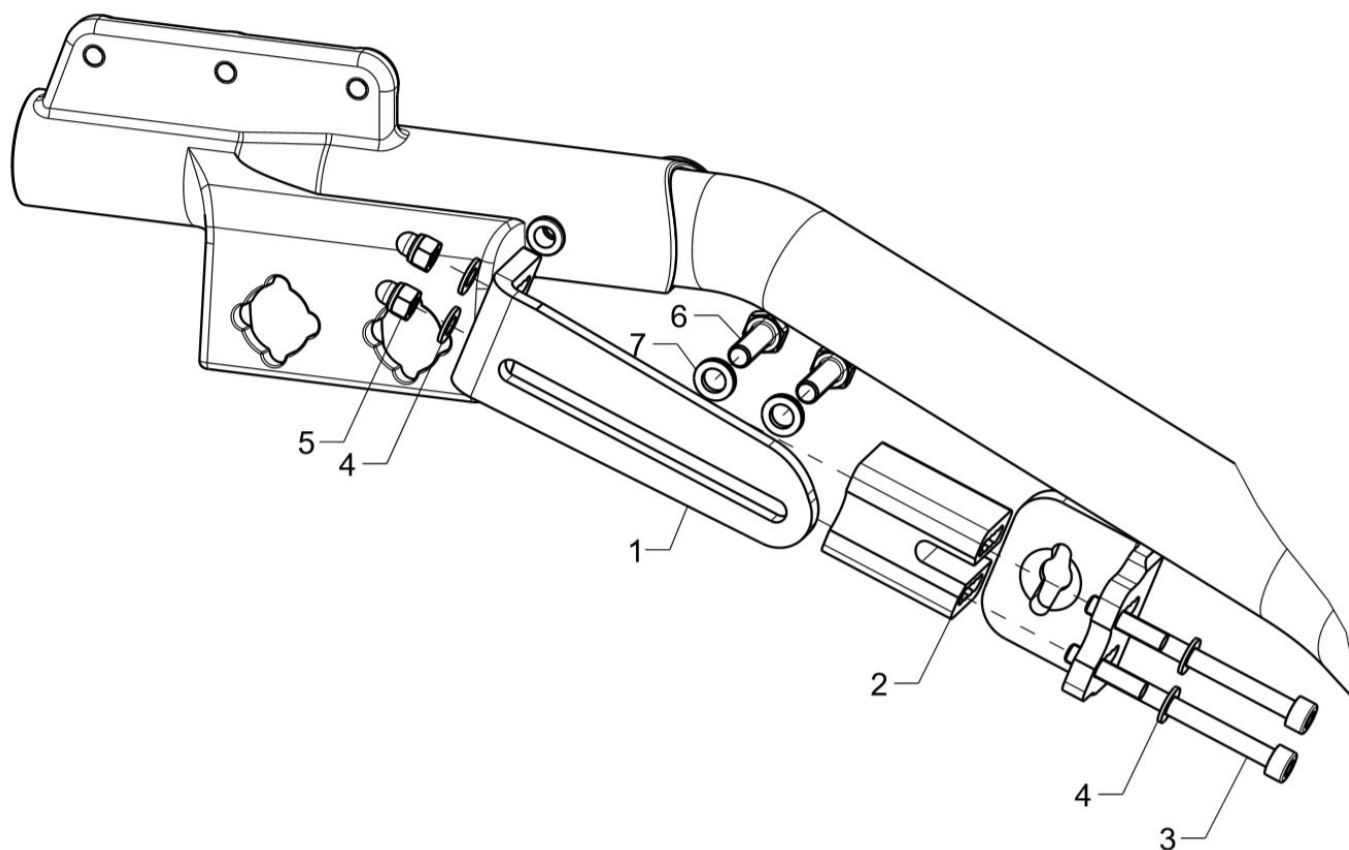

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	35351004			35.35.10.01.02.78
o.B.	1	35351003			35.35.10.01.01.78
2-7	1	35352002			35.35.20.02.00.78
3	2				
4	2		washer DIN 125 - A 5,3		02.61.05.31.01.20
5	2		hexagon cap nut low form with clamp DIN 986 - M5		02.33.00.60.21.20
6	2		lock washer M6 (Nord-Lock)		02.99.20.65.01.10
7	2				01.03.03.13.02.40

Companion locking brake

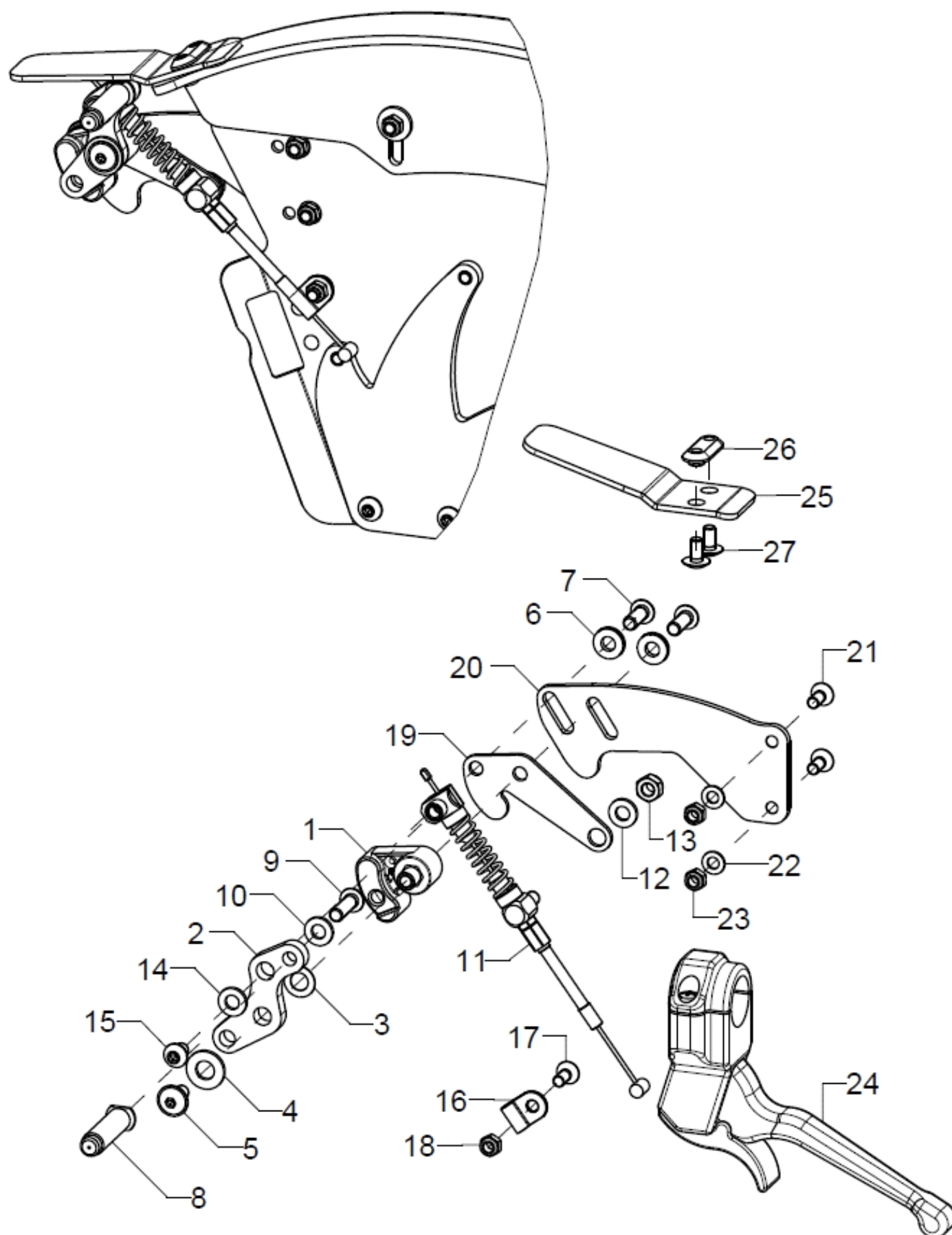




Spare parts list Mio Move

Brakes

Page 69 of 79


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-24	1	35360202			
o.B	1	35360201			
1-7+19	1	35360204			
o.B	1	35360203			
1	1				35.35.01.01.01.78
o.B.	1				35.35.01.01.02.78
2	1				35.34.01.05.00.00
3	1				02.68.08.03.00.30
4	1		washer DIN 125 - A 8,4 - PA black		02.61.08.41.61.71
5	1				01.43.08.48.62.61
6	2		lock washer M6 (Nord-Lock)		02.99.20.65.01.10
7	2		lens head screw with hexagon socket ISO 7380 - M6x16		01.42.08.52.65.00
8-10	1	352518	brake bolt 40 mm left-hand thread incl. mounting set		35.25.01.18.00.10
o.B.	1	352506	brake bolt milled 40 mm incl. mounting set		35.25.01.15.00.10
9	1		lens head screw with hexagon socket ISO 7380 , M6x20 left-hand thread		01.42.08.54.05.01
o.B.	1		lens head screw with hexagon socket ISO 7380 - M6x20		01.42.08.54.05.00
10	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
11-18	1	35360210			
12	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
13	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
14	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
15	1		lens head screw with hexagon socket ISO 7380 - M6x8		01.42.08.47.65.00
16-18	1	35360205			04.14.20.40.19.01
17	1		counter sunk screw with hexagon socket DIN 7991 M5x12		01.65.07.89.05.00
18	1		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
19	1	35350104			35.34.01.01.00.00
20-23	1	35360215			35.36.02.01.00.78
o.B	1	35360216			35.36.02.02.00.78
21	2		counter sunk screw with hexagon socket DIN 7991 M5x12		01.65.07.89.05.00
22	2		washer DIN 125 - A 5,3		02.61.05.31.01.20
23	2		hexagon nut self-locking low form DIN 985 - M5		02.32.00.50.01.20
24	1	352010	hand brake lever for attendant brake incl. mounting set		04.54.92.22.20.10
25-27	1	35360206			35.36.02.05.00.78
26	1				35.36.02.06.00.78
27	2				01.43.07.88.62.60

Cable for companion locking brake

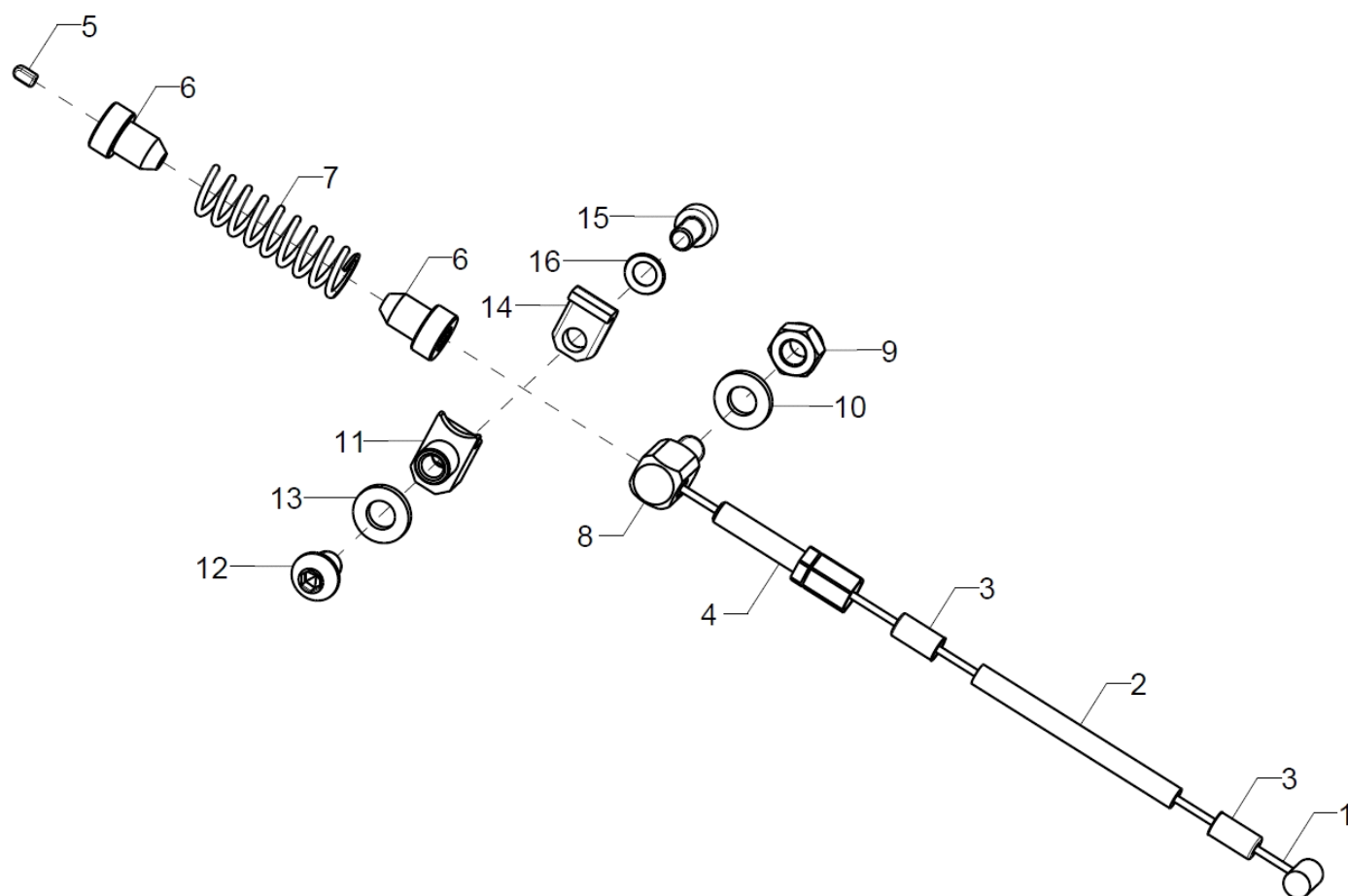


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-16	2	35360210			
1	1	352012	inner Bowden cable 1,5x2250		04.54.91.11.52.20
2	1	352013	outer shell for Bowden cable black 150mm		04.54.91.21.53.00
3	2	352412	end bush metal $\varnothing 5$		04.54.91.50.10.41
4	1	352406	setting screw for Bowden cable M5x34 mm		04.54.01.40.50.21
5	1	352005	end cap for inner Bowden cable $\varnothing 1-1,5$ mm		04.54.91.50.10.40
6	2	35340104			353401040000
7	1	35310508	pressure spring, 1,4x9,8x44,5		04.21.10.04.50.00
8-10	1	35340103			35.34.01.03.00.00
9	1		hexagon nut self-locking low form DIN 985 - M6		02.32.00.60.01.20
10	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
11-16	1	35360208			
12	1		lens head screw with hexagon socket ISO 7380 - M6x8		01.42.08.47.65.00
13	1		washer DIN 125 - A 6,4		02.61.06.41.41.20
14	1				35.36.03.03.00.78
15	1				01.23.07.87.05.00
16	1		lock washer S5		02.99.10.50.01.10

Brake lever for drum brake

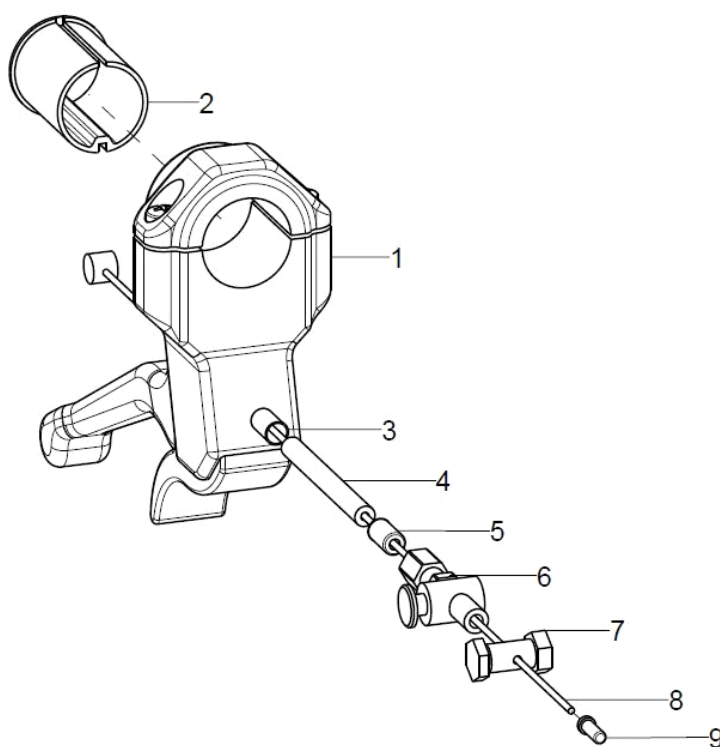


pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-9	2	352016	hand brake lever complete		XXXXXXXXXXXX
1+2	1	352010	hand brake lever for attendant brake incl. mounting set		04.54.92.22.20.10
2	1	352011			35.20.11.01.00.28
3-9	2	352006			35.20.06.99.00.10
3	1				04.54.91.50.11.21
4	1		outer shell for Bowden cable black 150mm		04.54.91.21.53.00
5	1		end bush metal ø5		04.54.91.50.10.41
6	1		set screw for Bowden cable M6x40 mm		04.54.01.40.60.20
7	1		clamp screw for bowden cable		04.54.11.30.00.20
8	1		inner Bowden cable1,5x2250		04.54.91.11.52.20
9	1		end cap for inner Bowden cable ø1-1,5 mm		04.54.91.50.10.40

Bumper

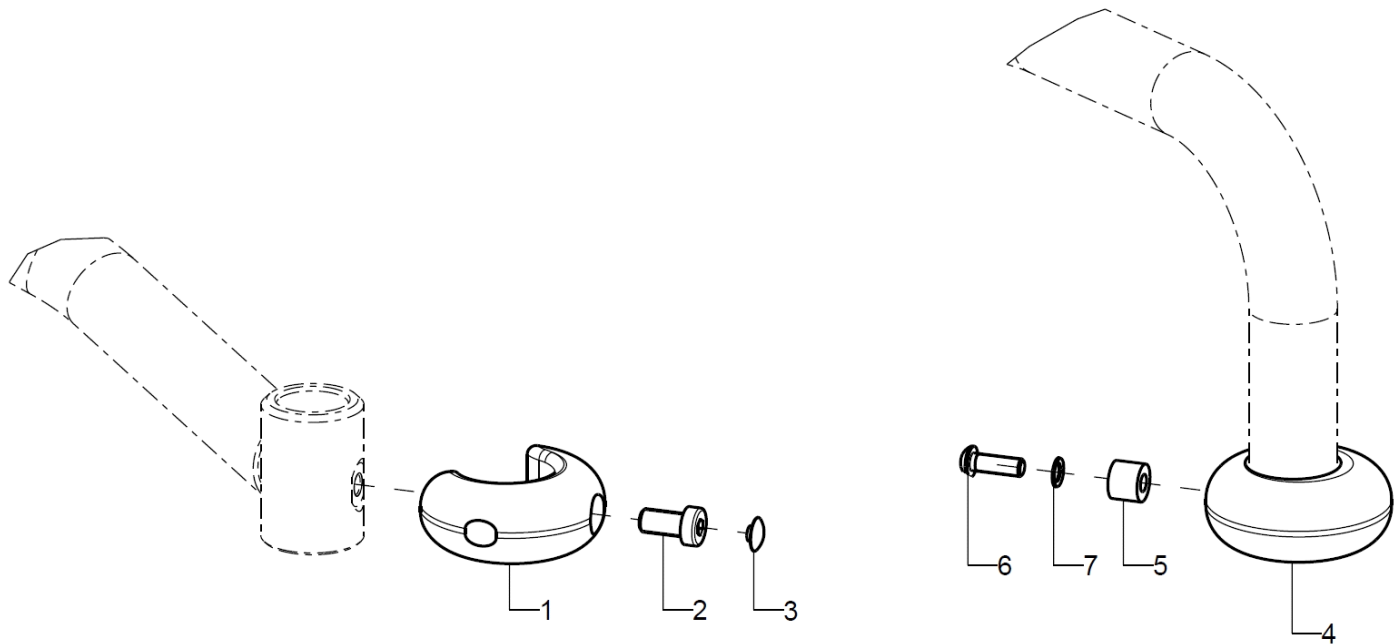

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1-3	2	910201			91.02.01.02.00.77
o.B.	2	910202			91.02.01.01.00.77
2	1				01.24.09.72.05.00
3	1				04.63.00.12.00.10
4-7	2	910203			91.02.01.03.00.77
5	1				91.02.01.05.00.77
6	1		lens head screw with hexagon socket ISO 7380 - M6x18		01.42.08.53.05.00
7	1		lock washer S6		02.99.10.60.01.10


Water bottle (without tipping lever)

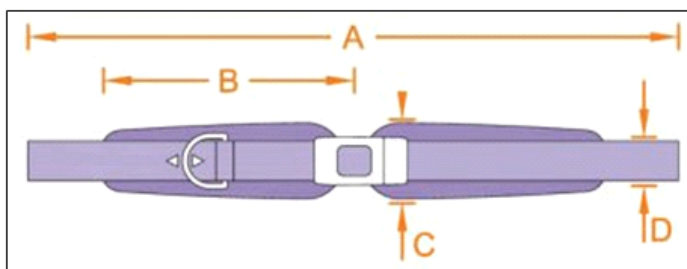

Größe	A	B	C	D
1	148	14	5	2,5
2	156	17	6	2,5

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	78050201			04.56.91.05.01.00
o.B.	1	78050202			04.56.91.05.02.00
o.B.	1	780107			78.01.06.02.01.00


2 point pelvic belt with metal fastener

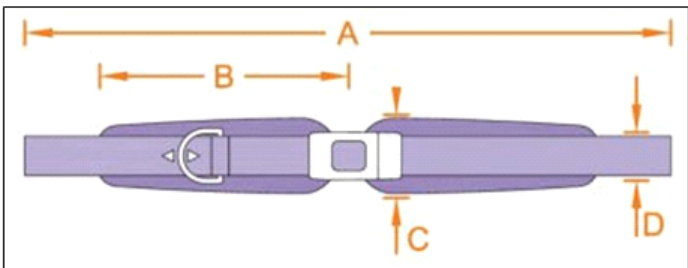

Größe	A	B	C	D
1	148	14	5	2,5
2	156	17	6	2,5

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	78050204			04.56.91.06.01.00
o.B.	1	78050205			04.56.91.06.02.00
o.B.	1	78050210			04.56.91.01.01.11
o.B.	1	780107			78.01.06.02.01.00


2 point pelvic belt with plastic fastener

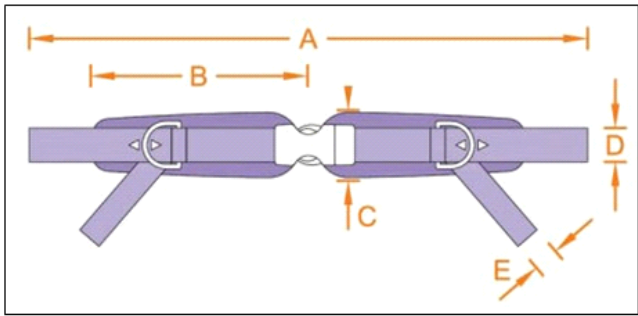

Größe	A	B	C	D	E
1	148	14	5	2,5	2,5
2	156	17	6	2,5	2,5

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	78050401			04.56.91.07.01.00
o.B.	1	78050402			04.56.91.07.02.00
o.B.	1	780107			78.01.06.02.01.00


4 point pelvic belt with metal fastener

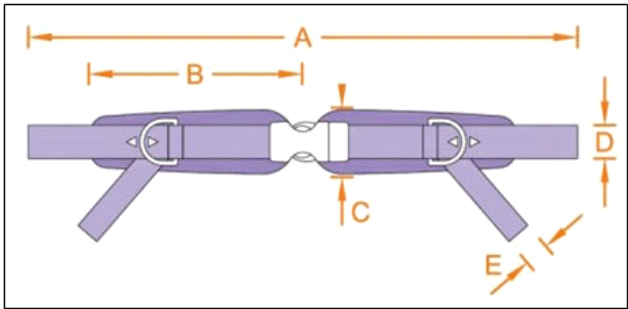

Größe	A	B	C	D	E
1	148	14	5	2,5	2,5
2	156	17	6	2,5	2,5

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	78050404			04.56.91.08.01.00
o.B.	1	78050405			04.56.91.08.02.00
o.B.	1	78050210			04.56.91.01.01.11
o.B.	1	780107			78.01.06.02.01.00


Chest shoulder belt

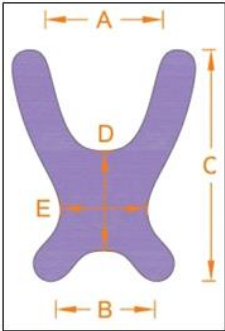



Größe	A	B	C	D	E
0	8	17	22	8	11
1	9	21	27	10	12
2	12	24	36	14	14

pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1	1	781000			04.56.92.05.00.00
o.B.	1	781001			04.56.92.05.01.00
o.B.	1	781002			04.56.92.05.02.00
o.B.	1	780107			78.01.06.02.01.00



Sun shade

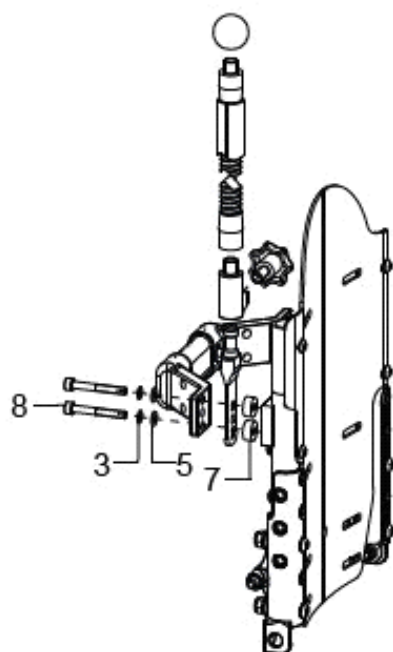

Sonnenschutzdach an Stabistange tief

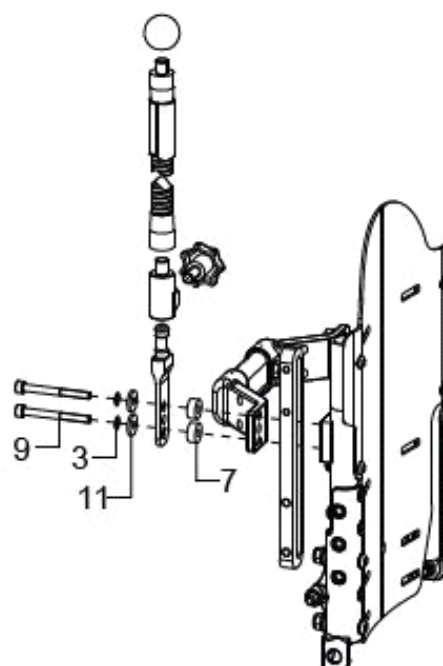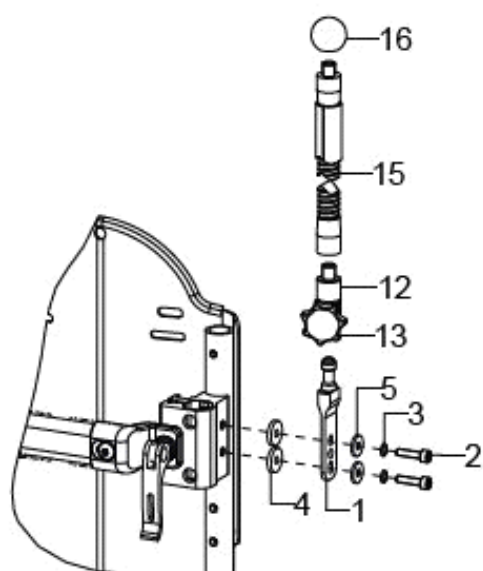
Sonnenschutzdach mit Pelotte
Stabistange tief


Sonnenschutzdach am Schiebegriffhalter

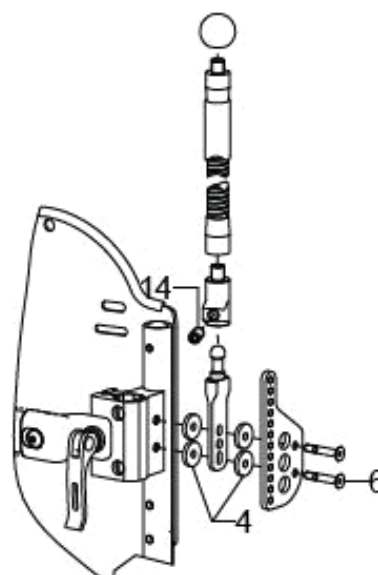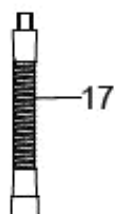
Sonnenschutzdach mit Pelotte
am Schiebegriffhalter

Verlängerung
Schwanenhals




pcs.	Q/1	item no.	Product name:	Variant:	drawing no.
1- 5+12+1 3+15+1 6+18	1				
1+4+6+ 7+12+1 4-16+18	1				
1+3+5+ 7+8+12 +13+15 +16+18	1				
1+3+7+ 9+13+1 5+16+1 8	1				
1	2	91030204			91.03.02.04.00.00
2	4		cylinder screw DIN 912 M5x20		01.22.07.94.72.40
3	4		lock washer S5		02.99.10.50.01.10
4	4				22.00.12.02.00.10
5	4		washer DIN 7349 - 5,3		02.77.05.22.00.20
6	4		counter sunk screw with hexagon socket DIN 7991 M5x30		01.65.07.98.05.00
7	4		bushing Ø14/5,2x6		40.55.00.01.00.76
8	4		cylinder screw DIN 912 M5x40		01.22.08.00.62.40
9	4				
11	4		washer DIN 7349 - 5,3		02.77.05.22.00.20
12	2	91030203			91.03.02.03.00.00
13	2		7-star knob ø32 - M8x15 external thread, black		04.05.47.14.15.40
14	2		elastic pressure piece M8		04.14.50.80.21.21
15-16	2	91030202			91.03.02.02.00.00
16	2				04.06.12.50.80.50
17	2	91030500			91.03.02.09.00.00
18	1	91030301			91.03.03.99.00.00